











Instructions for Use

ICEROSS SEAL-IN X LOCKING®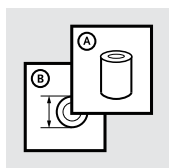

		3
		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	14
ES	Instrucciones para el uso	19
IT	Istruzioni per l'uso	24
NO	Bruksanvisning	29
DA	Brugsanvisning	33
SV	Bruksanvisning	37
EL	Οδηγίες Χρήσης	41
FI	Käyttöohjeet	46
NL	Gebruiksaanwijzing	50
PT	Instruções de Utilização	55
PL	Instrukcja użytkowania	60
CS	Návod k použití	65
TR	Kullanım Talimatları	69
RU	Инструкция по использованию	74
JA	取扱説明書	79
ZH	中文说明书	83
KO	사용 설명서	87
AR	تعليمات الاستخدام	92

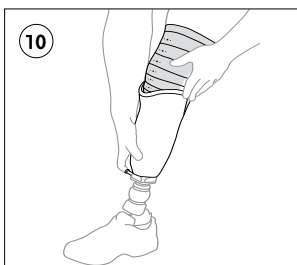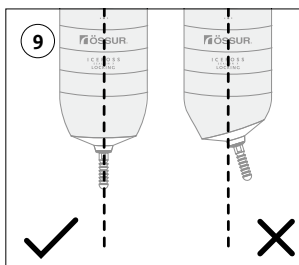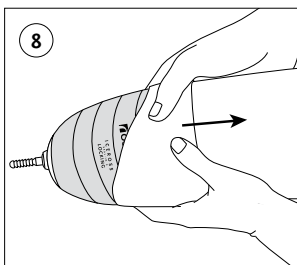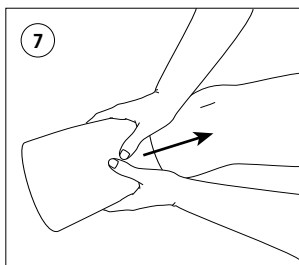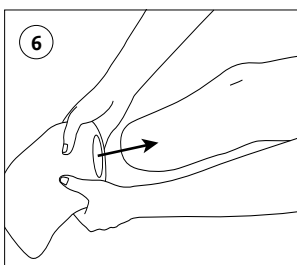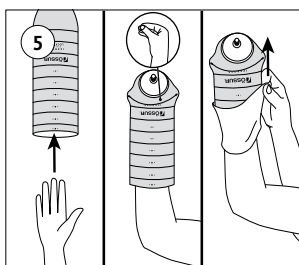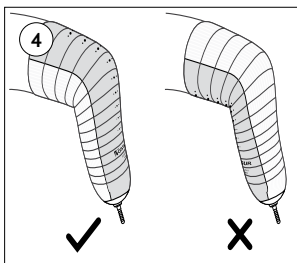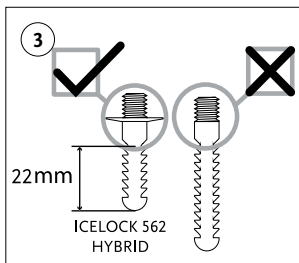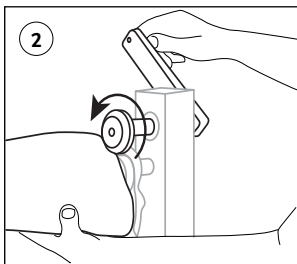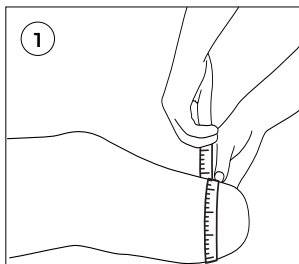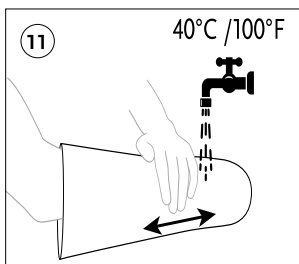ENGLISH

PRODUCT DESCRIPTION

The **Iceross Seal-In X Locking** liner is referred to as the “*liner*” in the following document. This document provides important information on the indications for use, fit and handling of the liner. This document is intended for a certified prosthetist and the user of the liner.

Liner Description

The Iceross Seal-In X Locking liner is a silicone liner for trans-tibial users. It is available with 3 and 6 mm profiles.

The liner is to be used with an Iceross Seal-In X Seal, Icelock 562 Hybrid, and an airtight socket. Combining these components will result in the residual limb being suspended by a mechanical distal attachment and a distal vacuum chamber created below the seal.

INDICATIONS FOR USE

Lower limb amputation.

NOTE: Users should have an appropriate level of physical fitness to use an external prosthesis for load transfer while ambulating.

CONTRAINDICATIONS

None known.

INTENDED USE

The Iceross Seal-In X Locking liner is a single patient, non-invasive and reusable prosthetic device designed for daily use by trans-tibial amputees with low to high impact levels.

The liner is only to be used for the exoprosthetic fitting of the residual limb.

GENERAL SAFETY INSTRUCTIONS

Warnings: Misuse of the liner may result in loss of suspension.

Do not use for individuals with cognitive limitations not allowing for safe use of the liner.

Precautions: In cases of functional change or loss, the user should immediately cease using the liner and contact a health practitioner. This may be the result of liner damage or changes in the residual limb.

REQUIRED COMPONENTS

Only use in combination with a suitable Iceross Seal-In X Seal, Icelock 562 Hybrid and an airtight socket.

Please refer to the “*Instructions for Use*” supplied with the Iceross Seal-In X Seal and Icelock 562 Hybrid.

LINER SIZE SELECTION

Precautions: Selecting the correct liner size is essential. Selecting a size which is too small/tight may result in pistoning, numbness, or blistering. Selecting a size which is too large/loose, may result in increased perspiration and movement of the limb inside the liner, which may lead to blisters and rashes. If any of the above symptoms, or any other indications of improper fit are experienced, the user should contact their

practitioner immediately.

To determine the correct size:

- Measure the circumference of the residual limb 4 cm from the distal end (**Fig. 1**).
- Choose the corresponding liner size. If the measurement taken does not have a corresponding liner size, round the measurement down.

NOTE: Do not choose a liner size exceeding the measurement, i.e. do not round up, e.g.: if the residual limb measures 24,5 cm at the indicated location, choose liner size 23,5, but not size 25.

FITTING INSTRUCTIONS

Trimming: The liner may be trimmed to increase range of motion, or for user preference (**Fig. 2**). The Iceross Curvemaster is specifically designed to produce a smooth, rounded edge when trimming, minimizing the risk of skin irritation and liner tears.

Precaution: Do not cut the liner below the proximal trim-lines of the socket. Excessive trimming may reduce the suction between the liner and limb, and compromise suspension.

Distal Attachment:

Precaution: Caution must be taken to properly tighten the attachment pin to the distal attachment.

Warnings:

- Always apply Loctite 410/411 or equivalent compound to the threads of the Össur attachment pin (used with Icelock 562 Hybrid) and torque to 4 Nm (3ft.lbs).
- Do not use flangeless pins with the liner (**Fig. 3**).

Donning:

Precautions:

- Failure to adhere to these instructions may compromise comfort and/or suspension.
- Do not apply lotion to the residual limb immediately prior to donning the liner. To prevent liner damage, the residual limb should be clean and dry.
- Open wounds or damaged skin should be covered with a bandage or other appropriate covering to prevent direct contact between the wound and liner.
- Ensure the inside of the liner is clean, dry, and free from any foreign objects that may cause skin irritation.

1. The liner should be oriented with the logo on the front of the residual limb (**Fig. 4**).
2. Grip the top of the liner from the inside and slide it over the hand until the inner surface is fully exposed, taking care not to damage it with fingernails (**Fig. 5**).
3. After exposing as much of the distal end of the liner as possible, position it against the distal end of the residual limb (**Fig. 6**).

NOTE: Make sure the pin is positioned centrally on the distal end of the residual limb (**Fig. 9**).

4. With light compression, roll the liner all the way upwards onto the limb (**Fig. 7, 8**), taking care not to damage it with fingernails. Do not tug or pull.

5. Check that no air pockets are present.

SOCKET CONSTRUCTION GUIDELINES

The following socket materials are recommended to ensure long-term liner function:

- All PETG materials
- Laminated sockets
- Thermoplastic inner socket materials with a smooth surface
- EVA inner socket materials

The inner surface of the socket must be airtight and as smooth as possible, irrespective of socket material.

To achieve the smoothest surface possible on the plaster positive model, it is recommended to use a fine abrasive for final smoothing, followed by silicone spray.

Using a soft/ smooth socket lining material will prolong the sealing function.

The Icelock 562 Hybrid is essential for liner function and must be laminated in the socket distally. Refer to Icelock 562 Hybrid “*Instructions for Use*” for details.

SOCKET DONNING

Precaution: Don an Iceross Seal-In X Seal ring before donning the socket. Do not use alcohol spray or other lotions with the Iceross Seal-In X Seal before donning the socket.

With the liner and seal ring correctly configured, simply step into the socket, expelling all air until attachment pin fully engages lock (**Fig. 10**).

SOCKET FIT

Achieving the intended function of the liner is dependent on the fit of the hard socket. Appropriate socket design is essential to achieve suspension, control, and comfort.

Follow the checklist to control socket fit:

- User can easily don the socket without the use of lubricant aids.
- Full contact between the Iceross Seal-In X Seal and the socket inner wall.
- No air pockets develop when the user loads the prosthesis.
- Good proximal fit.
- No pistoning in the socket.
- No visible movement when pulling on the prosthesis.
- Good rotational stability.
- Full movement possible without compromising distal vacuum.

Precaution: If feedback indicates any of the above conditions are not met, the socket will need to be adjusted or remade accordingly.

CARE AND CLEANING

NOTE: Daily cleaning of the residual limb is essential. Use of a mild, pH balance, 100% fragrance – and dye-free liquid soap is recommended. If skin is dry, apply a pH balance, 100% fragrance- and dye-free lotion to nourish and soften the skin.

Perspiration:

Precaution: If liner use results in excessive perspiration such that stability or suspension are compromised, it is recommended to remove the liner and dry off both the liner and the residual limb.

Cleaning: It is essential to take proper care of the liner and wash it daily following use. The liner can be used immediately after cleaning and drying.

- Remove the liner, turn inside out (**Fig. 5**).
- Wash with pH balanced, 100% fragrance- and dye-free liquid soap.
- The liner can also be machine washed (40°C/warm) with a mild detergent. Fabric softeners, bleaches, and other products/cleaning solutions may result in liner damage and should NOT be used.
- After washing, rinse the product thoroughly with hot water (**Fig. 11**), and pat dry on both sides with a lint-free cloth.

NOTE: Always return the liner to its neutral state (with the textile cover facing out) immediately after cleaning. Do not store inside-out as liner damage may result.

Inspection:**Precautions:**

- The liner should always be checked for wear and damage. Users are advised to contact their practitioner immediately if any damage is evident, as it may compromise liner effectiveness.
- Users should monitor skin health daily. In case of abnormal symptoms, the user is advised to discontinue use of the product and contact their practitioner immediately.
- Both the practitioner and user must ensure the liner is NOT exposed to glass, carbon fibers, or other foreign particles. Such substances can become embedded in the liner material causing skin irritation. Washing alone may not be sufficient to eliminate the problem. The liner should be returned to the practitioner if it is inadvertently exposed to foreign substances or chemicals.
- User should be aware that other common household or bath products may cause or contribute to skin irritation, e.g. soaps, deodorants, perfumes, abrasive cleaners, aerosol- or alcohol-sprays.

DISPOSAL

All components of the product and packaging should be disposed of in accordance with respective national environmental regulations. Users should contact their local governmental office for information on how these items can be disposed of in an environmentally sound manner.

WARRANTY

Össur offers a 6 month limited warranty on the liner from invoice date.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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