KOA[™] LP FOOT

PRODUCT HIGHLIGHTS:

ULTRA-LOW PROFILE WITH A FUNCTIONAL SANDAL TOE SPLIT-KEEL DESIGN DELIVERS INVERSION AND EVERSION WATERPROOF ADJUSTABLE HEEL RESPONSE

OVERVIEW

The Koa LP is a moderate to high activity fiberglass prosthetic foot providing both smooth rollover and energy return. With one of the lowest build heights among fiberglass feet, the Koa LP can be used with longer residual limbs and a range of components.

The foot's mutiaxial split keel design delivers excellent ground compliance on a variety of terrain and provides greater stability for those adjusting to a new prosthesis. The Koa LP foot is suitable for both transtibial and transfemoral amputees up to 365 pounds.







OTRANSFEMORAL

OTRANSTIBIAL

LCODE SUGGESTIONS: L5981, L5986

To select proper composite toe spring stiffness:

- 1. Calculate adjusted body weight. (Adjusted body weight = patient body weight + loads normally or routinely carried.)
- 2. Using adjusted body weight and patient activity level, select toe spring stiffness from the chart below.

	Impact Levels		
User Weight	Low	Medium	High
99-115 lbs (45-52 kg)	2	2	2
116-130 lbs (53-58 kg)	2	2	3
131-150 lbs (59-68 kg)	2	3	4
151-170 lbs (69-77 kg)	3	4	5
171-194 lbs (78-88 kg)	4	5	6
195-221 lbs (89-100 kg)	5	6	7
222-256 lbs (101-116 kg)	6	7	8
257-287 lbs (117-130 kg)	7	8	9
288-324 lbs (131-147 kg)	8	9	-
325-365 lbs (148-165 kg)	9	-	-

Item Numbers:

Ordering Example: Koo1-25L-2

Koa LP Foot, 25 cm, left foot, tan color, low toe resistance

Koo1- 25L-2

FS01-25LT


Model:	Size of Foot:	Foot:	Toe Resistance:
K001 = Koa LP Foot	23 to 28 cm	L = Left R= Right	2 - 9 (refer to chart above)

Size (cm):	Side of Foot:	Color:
23 to 28 cm	L = Left	B = Buff
	R = Right	M = Medium Brown
		T = Tan

KOA™ LP FOOT: SPECIFICATIONS

HEEL HEIGHT: 10 mm (3/8")

HEEL WEDGES: Medium (red), Firm (blue)

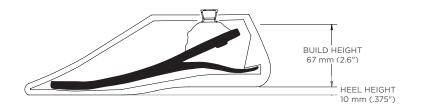

APPROXIMATE PRODUCT WEIGHT:

SIZE	APPROXIMATE WEIGHT WITH SHELL
23 - 25 cm	465 g
26 -28 cm	629 g

TRIAL PERIOD: 60 days from date of patient fitting

WARRANTY: Foot: 36 months from date of patient fitting

Foot Shell: 6 months from date of patient fitting

WHAT'S INCLUDED:





700-SS001

ADDITIONAL OPTIONS:



Koa LP Sandal Toe Foot Shell (sold separately) Buff (shown), Tan, Medium Brown FS01-XXXX