INTRODUCING

Willow Wood KOAMLP





NOW AVAILABLE!

The Koa LP is a moderate to high activity fiberglass prosthetic foot providing both smooth rollover and energy return. With one of the lowest build heights among fiberglass feet, the Koa LP can be used with longer residual limbs and a range of components.

KEY FEATURES:

- Ultra-low profile
- Split keel and heel delivers greater ground compliance
- Suitable for transfemoral and transtibial amputees
- Adjustable heel response



Part number includes foot module, foot sock, and 2 heel wedges.

Koa LP foot shell sold separately.

* To select proper composite toe spring stiffness:

1. Calculate adjusted body weight.

(Adjusted body weight = patient body weight + loads normally or routinely carried.)

2. Using adjusted body weight and patient activity level, select toe spring stiffness from the chart below.

	IMPACT LEVELS —		
User Weight	Low	Medium	High
99-115 lbs (45-52 kg)	2	2	2
116-130 lbs (53-58 kg)	2	2	3
131-150 lbs (59-68 kg)	2	3	4
151-170 lbs (69-77 kg)	3	4	5
171-194 lbs (78-88 kg)	4	5	6
195-221 lbs (89-100 kg)	5	6	7
222-256 lbs (101-116 kg)	6	7	8
257-287 lbs (117-130 kg)	7	8	9
288-324 lbs (131-147 kg)	8	9	-
325-365 lbs (148-165 kg)	9	-	-

SPECIFICATIONS

L-Codes:	L5981, L5986	
Sizes:	23 - 28 cm	
Product Weight:	585 g**	
Weight Limit:	165 kg (365 lb)	
Build Height:	67 mm (2.6")	
Heel Height:	10 mm (3/8")	
Heel Wedges:	Medium (red), Firm (blue)	
Foot Shell:	Sandal toe	
Trial Period:	60 days	
Warranty:	36 months (foot), 6 months (shell)	
** Based on size 26 cm, category 4, with foot shell.		

FOR MORE INFORMATION CALL 800.848.4930 OR 740.869.3377 OR VISIT WILLOWWOODCO.COM.