

HOW TO MEASURE FOR AND APPLY THIS RIB BELT

CIRCUMFERENCE	
MEASURE AROUND THE CHEST AT THE BASE OF THE STERNUM	
UNIVERSAL REGULAR	29" - 38" (74 - 96.5 cm)
UNIVERSAL LARGE	39" - 48" (99 - 122 cm)

PRODUCT NO. 2659

ELASTIC RIB BELT FOR WOMEN

LATEX FREE

DEGREE OF SUPPORT

MINIMUM •

INDICATIONS

Compression and support for rib fractures and contusions around the rib cage

MEASURING INSTRUCTIONS

A. Measure around the chest at the base of the sternum

APPLICATION INSTRUCTIONS

1. Apply in a standing position.
2. Position belt over rib area.
3. Pull the side with the loop fastener until snug, then overlap the hook fastener on top of it.
4. Readjust for comfort.

