Instructions for Use

4-PRONG SOCKET ADAPTER JUNIOR
3-PRONG SOCKET ADAPTER SS
4-PRONG SOCKET ADAPTER SS

EN		Instructions for Use	3
DE		Gebrauchsanweisung	8
FR		Notice d'utilisation	14
ES		Instrucciones para el uso	20
IT		Istruzioni per l'uso	26
NO		Bruksanvisning	32
DA		Brugsanvisning	37
SV		Bruksanvisning	42
EL		Οδηγίες Χρήσης	47
FI		Käyttöohjeet	53
NL		Gebruiksaanwijzing	58
PT		Instruções de Utilização	64
PL		Instrukcja użytkowania	70
TR		Kullanım Talimatları	76
RU		Инструкция по использованию	82
JA		取扱説明書	89
ZH		中文说明书	94
KO		사용 설명서	99

INTENDED USE

The device is intended as a connection component of a prosthetic system that replaces a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

For more detailed device information, see **Table 1**.

Indications For Use and Target Patient Population

- Lower limb loss, amputation, or deficiency
- No known contraindications

For weight limit and impact level, see **Table 2**.

GENERAL SAFETY INSTRUCTIONS

Warning: Use of a lower limb prosthetic device carries an inherent risk of falling which may lead to injury.

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

Warning: Only use Össur replacement parts for the device for ensured durability.

The device is for single patient use.

ASSEMBLY INSTRUCTIONS

Instructions for integration into laminated sockets:

Note: It is recommended that the socket is laminated in two stages.

Caution: When shaping the pronged adapter, care must be taken to prevent distortion of the distal threads.

1. Isolate cast with PVA bag.
2. Apply appropriate layup material for first lamination.
3. Apply single 50mm carbon tapes crossed over distal end of cast, in AP/ML direction.
4. Apply the outer PVA bag and laminate.
5. Once cured, remove PVA bag and roughen outer surface of lamination.
6. Attach the Adapter in correct alignment and rotation with putty.
7. Position the Lamination Cap on Adapter and secure it by locking the screw.
8. Apply clay to Pinch Bolt and Pinch Bolt housing to protect from resin.
9. Tie 50mm carbon tapes into grooves on Adapter, position in AP/ML direction.
10. Apply layup material and combinations of material appropriate to the individual's weight and impact level.
11. Layers of material may be trimmed to different lengths to provide a thinner lighter proximal socket.
12. Tie 50mm carbon tapes into grooves on Adapter, position in AP/ML direction.
13. Continue with appropriate layup material.
14. Laminate with 80:20 acrylic lamination resin.

Note: Remove the Lamination Cap after lamination by unscrewing from Adapter.

Warning: For rotation adjustment always engage threads as far as possible whilst maintaining correct rotation of adapter and torque pinch bolt.

Warning: For secure fastening use medium strength threadlocker and apply rated torque twice, see **Table 1**.

USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

Environmental Conditions

See which environmental conditions apply to the device in **Table 1**.

A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e. g., rain), no submersion is allowed.

No contact with salt water or chlorinated water is allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

A Waterproof device can be used in a wet or humid environment and submerged in up to 3-meter-deep water for a maximum of 1 hour.

It can tolerate contact with: Salt water, chlorinated water, perspiration, urine, and mild soaps.

It can also tolerate occasional exposure to sand, dust, and dirt.

Continuous exposure is not allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water after exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

MAINTENANCE

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

SPECIFICATIONS

Table 1 - Device Specifications

Part #	Description	Material	Tightening Torque	Environmental Conditions
			Pinch Bolt	
AJ-114040	4-Prong Socket Adapter	Stainless Steel	5 Nm	Weatherproof
AJ-114140	Junior			Waterproof
A-114030	3-Prong Socket Adapter SS			
A-114040	4-Prong Socket Adapter SS			

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur prosthetic devices are designed and verified to be safe and compatible in combination with each other and custom-made prosthetic sockets with Össur adapters, and when used in accordance with their intended use.

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Compliance

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity, this may correspond to 3-5 years of use.

Table 2 - Usage Specifications

Part #	Impact Level	Weight Limit	ISO 10328 - Label Text
AJ-114040	The device is intended for children and adolescent patients.	45 kg	ISO 10328 - P3 - 45 kg
AJ-114140		60 kg	ISO 10328 - P4 - 60 kg
A-114030	The device is for low to high impact use, e.g., walking and occasional running.	166 kg	ISO 10328 - P7 - 166 kg
A-114040			

ISO 10328 - “P” - “m”kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use
see manufacturer's written instructions on
intended use!

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