50R320N Cyberspine TLSOx4

EN Instructions for Use2

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50R320N Cyberspine TLSOx4

Thank you for using this Ottobock device. The following information will help you fit and care for this device. Please review it before using this device and ask your physician, healthcare provider, or practitioner for clarification if needed.

Indications for Use:

Indicated as an adjunct to pre and post operative procedures, for acute pain relief by thoracic postural extension alignment, early onset through variable stages of osteoporosis, compression fractures of the thoracic spine, kyphosis, and thoracic mechanical back pain.

Warning

This device does not prevent injury and is not intended to reduce or eliminate the risk of injury.

Warning

Always consult with your physician or healthcare provider before making changes to the device.

Warning

Carefully read all instructions for use before using or fitting this device.

Warning

Proper rehabilitation and activity modification are also a part of a safe treatment program.

Warning

If you experience pain, swelling, or any unusual reaction, contact your physician or healthcare provider immediately.

CAUTION

This device is offered under order of a physician or other qualified healthcare provider.

CAUTION

This device is intended for single patient use and is not intended to be reused on a second patient.

Application Instructions

Please follow all prescribing physician's orders for frequency and duration of use.

1. Locate the D ring handle on the left or right side of the device. Once located, fasten the handle to the device where the pulley system cables are lax and the brace can be fitted without interference.
2. While standing in an upright position, hold the side panels apart, and center the back panel of the device along the spine just above the tailbone.
3. Wrap the left side panel to the front to position the front panel. Then wrap the right side panel on top of the left side panel to close. Attach both sides in front. Adjust the front or back panels as needed.
4. Loosen the shoulder straps and connect to the front loop. Tighten as needed.
5. Adjust the anterior thoracic extension so the top edge of the sterna pad is about 1" below the sternal notch. Check position while patient is sitting for optimal length without impingement.
6. Pull the D ring handle out and away from the pulley unit to reach desired compression and support. Place the handle on the fabric fastener band securely.

Cleaning and Care

Remove the rigid components. The rigid components can be wiped clean with a damp cloth. Softgoods can be hand washed in warm water (90°F–110°F or 32°C–43°C) with mild detergent. Triple rinse, squeeze, and air dry. Avoid twisting the material to remove water.

Liability/Warranty

This device is warrantied for a period of 60 days. The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use. Details of the warranty are also available at www.ottobockus.com.

Manufacturer's Limited Warranty

Otto Bock HealthCare, LP (Ottobock) warrants all of its devices, to the original purchaser, to be free from defects in materials and workmanship. This warranty applies, subject to normal wear and tear, when the devices are used as intended, without unapproved modifications, following all Ottobock instructions and requirements; and when they are fitted by or under the direct supervision of certified/licensed practitioners. This Limited Warranty does not cover device damage caused by accidents, neglect, misuse or operation beyond capacity, parts damaged by improper installation, substitution of parts not approved by Ottobock, or any alteration or repair by others that, in Ottobock's judgment, materially or adversely affect the device.

The duration of this Limited Warranty varies by product types and is effective from the date of delivery to the end-user. Please refer to www.ottobockus.com or call 800 328 4058 for questions. Ottobock's sole obligation under this Limited Warranty shall be to repair, replace, refabricate the device at no charge, or refund the cost of the device to the original purchaser, at Ottobock's sole discretion.

THE EXPRESS WARRANTIES SET FORTH ABOVE ARE IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND ALL SUCH OTHER WARRANTIES ARE HEREBY DISCLAIMED AND EXCLUDED BY OTTOBOCK. IN NO EVENT SHALL OTTOBOCK'S LIABILITY OF ANY KIND INCLUDE ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF OTTOBOCK SHALL HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH POTENTIAL LOSS OR DAMAGE.

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