Tube clamp adapters

DE Gebrauchsanweisung	3
EN Instructions for use	7
FR Instructions d'utilisation.....	11
IT Istruzioni per l'uso	15
ES Instrucciones de uso	20
PT Manual de utilização.....	24
NL Gebruiksaanwijzing.....	28
SV Bruksanvisning	33
DA Brugsanvisning	37
NO Bruksanvisning	41
FI Käyttöohje.....	45
PL Instrukcja użytkowania.....	49
HU Használati utasítás	53

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INFORMATION

Last update: 2017-05-22

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

1.1 Construction and Function

Tube clamp adapters are used as components for modular lower limb prostheses. Together with the tube adapter they connect the prosthetic foot to the proximal components. Adapter combinations allow for controlled angle and length adaptation in the sagittal and frontal plane as well as adjustment of inward and outward rotation. These instructions for use apply to the following tube clamp adapters:

Diameter	Reference number
30 mm	4R21, 4R21=1, 4R52, 4R52=1, 4R56=*, 4R69*
34 mm	4R82, 4R82=P, 4R91, 4R156=*, 4WR95=3

1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

2 Intended use

2.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

2.2 Area of application

Product approved for max. body weight

- The maximum approved body weight is specified in the technical data (see Page 10).

Additional constraints

- **4R21*, 4R52*:** Only for use in TF prostheses, below the prosthetic knee joint.
- **4R56*, 4R156*:** Only for use in hip disarticulation prostheses, above the prosthetic knee joint.

2.3 Environmental conditions

Allowable environmental conditions

Temperature range for use: -10 °C to +60 °C

Allowable relative humidity 0 % to 90 %, non-condensing

Unallowable environmental conditions

Mechanical vibrations or impacts

Perspiration, urine, fresh water, salt water, acids

Dust, sand, highly hygroscopic particles (e.g. talcum)

4WR95=3

Allowable environmental conditions

Temperature range for use: -10 °C to +60 °C

Storage and transport conditions: -10° C to +60° C, relative humidity: no restrictions

Allowable environmental conditions

Humidity: fresh water, salt water – cleaning and drying is required after contact and use.

Allowable immersion depth: 1 m

Unallowable environmental conditions

Sauna, extreme watersports

Mechanical vibrations or impacts

Perspiration, urine, acids or bases

Dust, sand, highly hygroscopic particles (e.g. talcum)

2.4 Service Life

In general, all modular adapters are tested by the manufacturer for 3 million load cycles. Depending on the patient's level of activity, this corresponds to a service life of 3 to 5 years.

3 Safety

3.1 Explanation of warning symbols

Warning regarding possible risks of accident or injury.

Warning regarding possible technical damage.

3.2 General safety instructions

Excessive strain on the product

Risk of injury due to breakage of load-bearing components

- ▶ Use the product according to the specified area of application (see Page 7).

Unallowable combination of prosthetic components

Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved for that purpose.
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

Use under unallowable environmental conditions

Risk of injury due to damage to the product

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).

Exceeding the service life

Risk of injury due to change in or loss of functionality and damage to the product

- ▶ Ensure that the approved service life is not exceeded.

Mechanical damage to the product

Risk of injury due to change in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).

- Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Tube clamp adapter	–
For 4R21, 4R21=1, 4R52, 4R52=1, 4R56*		
4	Set screw	506G3=M8x12-V
For 4R69*, 4R82, 4R91, 4R156*, 4WR95=3		
4	Set screw	506G3=M8x14
Spare parts/accessories (not included in the scope of delivery)		
Designation		Reference number
Single component pack for 4R156*		4D28
Consisting of: Cap screw, clamping bolt, clamping bolt with interior thread, plastic ring		

5 Preparation for use

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- Observe the alignment and assembly instructions.

Improper assembly of the screw connections

Risk of injury due to breakage or loosening of the screw connections

- Clean the threads before every installation.
- Apply the specified torque values.
- Follow the instructions regarding the length of the screws and about how to secure the screws.

5.1 Mounting the adapter

Incorrect mounting of the tube

Risk of injury due to breakage of load-bearing components

- Slide the tube all the way to the stop in the intended prosthetic component when mounting.

- > **Required materials:** 710D4 torque wrench, degreasing cleaner (such as 634A58 isopropyl alcohol)
- 1) Loosen the tube clamp cap screw by **2 turns**.
- 2) **For final assembly:** Degrease the insertion range of the tube clamp adapter and tube adapter.
- 3) Fully insert the tube adapter into the tube clamp adapter up to the stop. Align the clamping slot:
Tube clamp adapter: **anterior**
Tube clamp adapter, sliding: **anterior** or **medial**
- 4) Tighten the cap screw on the tube clamp, choosing the torque according to the thread diameter (M4: **5 Nm**, M5: **10 Nm**, M6: **13 Nm**).

5.2 Installation in the modular prosthesis

- > **Required tools and materials:**

710D4 torque wrench, 636K13 Loctite®

- Install the other prosthetic components as described in the respective instructions for use.

6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

7 Maintenance

- ▶ A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

8 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by

the manufacturer with sole responsibility according to Annex VII of the directive.

9.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

10 Technical data

Tube diameter: 30 mm

Reference number	4R21	4R21=1	4R69	4R69=AL
Weight [g]	130	125	75	75
System height [mm]	33			
Material	Stainless steel		Aluminium	
Max. body weight [kg]	100	136	125	136

Reference number	4R52	4R52=1	4R56	4R56=1	4R56=2
Weight [g]	75	80	85	85	100
System height [mm]	33		34		35
Material	Titanium				
Angling °	–	–	10	20	30
Max. body weight [kg]	100	136	100		

Tube diameter: 34 mm

Reference number	4R82	4R82=P	4R91
Weight [g]	95	90	140
System height [mm]	33	-12	33
Material	Titanium		Stainless steel
Max. body weight [kg]	150		

Reference number	4R156	4R156=1	4R156=2	4WR95=3
Weight [g]	140	165	175	130
System height [mm]	36	37	38	33
Material	Titanium			
Angling [°]	10	20	30	–
Max. body weight [kg]	150			

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Otto Bock HealthCare GmbH