2R40=*, 2R41=*, 2R48

DE	Gebrauchsanweisung	3
EN	Instructions for use	9
FR	Instructions d'utilisation	16
IT	Istruzioni per l'uso	23
ES	Instrucciones de uso	30
PT	Manual de utilização	37
NL	Gebruiksaanwijzing	44
SV	Bruksanvisning	51
DA	Brugsanvisning	57
NO	Bruksanvisning	63
FI	Käyttöohje	69
PL	Instrukcja użytkowania	76
HU	Használati utasítás	83
CS	Návod k použití	89
RO	Instrucțiuni de utilizare	96
HR	Upute za uporabu	103
SL	Navodila za uporabo	109
SK	Návod na používanie	115
BG	Инструкция за употреба	122
TR	Kullanma talimatı	129
EL	Οδηγίες χρήσης	135
RU	Руководство по применению	143
ZH	使用说明书	150
KO	사용 설명서	156

1

2

INFORMATION

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INFORMATION

Last update: 2015-11-09

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

1.1 Construction and Function

The 2R40=1 and 2R40=2 foot adapters for children are used to connect a SACH foot for children or a dynamic foot for children to the distal connector of the modular prosthesis. The 2R41=1, 2R41=2 tube adapters are used as

components for modular lower limb prostheses. They connect the prosthetic foot to the proximal components. The 2R48 tube adapter has an angled connector and is used to join the prosthetic knee joint to the prosthetic hip joint.

1.2 Combination possibilities

This prosthetic component is part of the Ottobock modular system for children. As a result of its design, it may be combined with other prosthetic components of the modular system for children. You can find possible restrictions in this section. Contact the manufacturer with questions.

2 Application

2.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

2.2 Area of application

The product was developed for use in a children's prosthesis.

- **2R40=1, 2R41=1, 2R48:** Approved for a body weight up to **max. 45 kg**.
- **2R40=2, 2R41=2:** Approved for a body weight up to **max. 35 kg**.

2.3 Environmental conditions

Allowable environmental conditions
Temperature range for use: -10 °C to +60°C
Allowable relative humidity 0 % to 90 %, non-condensing
Unallowable environmental conditions
Mechanical vibrations or impacts
Perspiration, urine, fresh water, salt water, acids
Dust, sand, highly hygroscopic particles (e.g. talcum)

3 Safety

3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

3.2 General safety instructions

 **CAUTION**

Excessive strain on the product

Risk of injury due to breakage of load-bearing components

- Use the prosthetic components according to the classification (see section "Area of application").

CAUTION

Unallowable combination of prosthetic components

Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved according to the section "Combination possibilities".
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

CAUTION

Use under unallowable environmental conditions

Risk of injury due to damage to the product

- ▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental conditions").
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).

CAUTION

Mechanical damage to the product

Risk of injury due to change in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Adapter	–
Only for 2R41=*, 2R48 tube adapter		
4	Set screw	506G3=M6x10

Spare parts/accessories (not included in the scope of delivery)		
Quantity	Designation	Reference number
Special adhesive with hardener		636W28
		636W18 and 636W19
Single component pack for 2R40=1 consisting of: 1 cap screw, 1 flat washer		2D6=M8
Single component pack for 2R40=2 consisting of: 1 cap screw, 1 flat washer		2D6=M6

5 Preparation for use

CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- Observe the alignment and assembly instructions.

CAUTION

Improper assembly of the screw connections

Risk of injury due to breakage or loosening of the screw connections

- Clean the threads before every installation.
- Apply the specified torque values.
- Follow the instructions regarding the length of the screws and about how to secure the screws.

5.1 Installing the foot adapter

- > **Required tools and materials:** Grinding tool, 710D4 torque wrench, degreasing cleaning agent (e.g. 634A3 acetone), special adhesive and hardener (see Page 12, table of spare parts/accessories)
- 1) For prosthetic feet with wooden components, roughen the connection surface and remove the grinding dust.

- 2) **INFORMATION: The foot adapter has to be glued to the prosthetic foot. Use the specified special adhesive for this purpose.**

Mix the special adhesive and hardener well together and apply to the connection surface.

- 3) Clean any soiling (e.g. caused by grease) on the metal surface of the foot adapter to be glued with a degreasing cleaning agent.
- 4) Position the foot adapter on the prosthetic foot, insert the screw and washer from below and tighten it (installation torque: **2 to 4 Nm**).
- 5) After **3 to 4 hours**, tighten the screw connection with the required installation torque (see table).
- 6) Leave the glued surface to harden (**16 hours at 20 °C** – the time increases for lower temperatures).

Reference number	Prosthetic foot size [cm]	Torque [Nm]	
		Prosthetic feet with wooden components	Prosthetic feet with plastic components
2R40=2	12-17	10	–
2R40=1	18-21	20	–

5.2 Adapting the tube adapter

CAUTION

Incorrect processing of tube

Fall due to damage to the tube

- Do not clamp the tube into a vice.
- For shortening the tube, use only a tube cutter.

CAUTION

Incorrect mounting of the tube

Fall due to breakage of load-bearing components

- Slide the tube all the way to the stop in the intended prosthetic component when mounting.

> **Recommended tools and materials:**

710D4 torque wrench, 719R3 tube cutter, 718R1 tube deburrer, degreasing cleaning agent (e.g. 634A3 Acetone)

- 1) Use the tube cutter to shorten the tube according to the patient's measurements.
- 2) Deburr the inside and outside of the cut edge with the tube deburrer.

5.3 Installation in the modular prosthesis

> **Recommended tools and materials:**

710D4 torque wrench, 636K13 Loctite®

- 1) Connect the connecting component to the tube adapter as described in the connecting component's instructions for use.
- 2) Position the pyramid receiver of the tube adapter distally in the prosthesis.
- 3) Align the clamping slot of one tube clamp adapter:
Tube clamp adapter: **anterior**
Tube clamp adapter, sliding: **anterior** or **medial**

Connecting the pyramid adapter and pyramid receiver

The pyramid adapter is fixed with the set screws of the pyramid receiver.

> **Required tools and materials:**

710D4 torque wrench, 636K13 Loctite®

- 1) **For definitive mounting:**
Use Loctite® to secure the set screws.
- 2) Screw in the set screws.
- 3) Use the torque wrench to tighten the set screws (2R41=1: **9 Nm**, 2R41=2, 2R48: **7 Nm**).

Alignment

The set screws in the pyramid receiver can be used to make static adjustments during alignment, trial fittings and after the prosthesis is finished.

Replacement and disassembly

The set position of the prosthetic component can be maintained during replacement or disassembly. In order to do this, unscrew the two set screws that are screwed in the furthest and located next to each other.

6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

7 Maintenance

- ▶ The prosthetic components should be inspected after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

8 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

9.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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Otto Bock HealthCare GmbH