Spots (616S134), Sticky Spots (616S132)

EN	Instructions for use	2
DE	Gebrauchsanweisung	4
FR	Instructions d'utilisation	7
IT	Istruzioni per l'uso	10
ES	Instrucciones de uso	13
PT	Manual de utilização	15
NL	Gebruiksaanwijzing	18
SV	Bruksanvisning	20
EL	Οδηγίες χρήσης	23
JA	取扱説明書	26
ZH	使用说明书	28

INFORMATION

Last update: 2016-08-19

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

Explanation of Symbols

English

 **CAUTION** Warnings regarding possible risks of accident or injury.

 **INFORMATION** Additional information on the fitting / use.

1 Description

1.1 Indications for use

The Spots 616S134 have been designed as volume management tool for prosthetic sockets. The Sticky Spots 616S132 have been designed for weight bearing and/or pressure relief in prosthetic sockets or orthoses (e.g. AFO, TLSO). Both types of Spots are an ideal tool where the skin is vulnerable to pressure.

1.2 Safety instructions

 **CAUTION**

Skin or tissue damages due to direct use of Spots or Sticky Spots on the skin. Uncovered PUR material is very sticky. Always apply Spots with the sticky side to the socket, liner or orthosis. Never use directly on the skin!

1.3 Function

Both types of Spots are made of polyurethane (PUR) which has unique flow and recover characteristics. Thereby they provide pressure relief or can fill distal gaps in prosthetic sockets (if used for volume management).

2 Handling

2.1 Spots 616S134

Place the Spot in distal portion of socket. If you are using a socket with an air expulsion valve in that area, a sheath or sock must be on the outside of the Spot to allow air to flow through the valve.

2.2 Sticky Spots 616S132

Sticky Spots have an adhesive on one side and include an oversized adhesive Lycra® cover. Simply clean the area for the Spot, remove backing, and stick it in place. Cover with Lycra® cover to keep it in place. Can be used in plastic applications and laminations.

INFORMATION

If used as fixed placement with sockets or orthoses proceed as follows:

- a) Attach the spot at desired location on positive plaster model.
- b) Pull plastic or laminate over it.
- c) Replace with Sticky Spot for final fitting.

3 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

3.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

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This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

3.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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Ottobock HealthCare GmbH