6R8

6R18

6R8, 6R18

DE	Gebrauchsanweisung	2
EN	Instructions for use	3
FR	Instructions d'utilisation	5
IT	Istruzioni per l'uso	6
ES	Instrucciones de uso	8
PT	Manual de utilização	9
NL	Gebruiksaanwijzing	11
SV	Bruksanvisning	13
DA	Brugsanvisning	14
NO	Bruksanvisning	16
PL	Instrukcja użytkowania	17
HU	Használati utasítás	19
CS	Návod k použití	20
TR	Kullanma talimatı	22
EL	Οδηγίες χρήσης	23
RU	Руководство по применению	25
JA	取扱説明書	27
ZH	使用说明书	28

INFORMATION

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INFORMATION

Last update: 2016-06-10

- Please read this document carefully before using the product.
- Follow the safety instructions to avoid injuries and damage to the product.
- Instruct the user in the proper and safe use of the product.
- Please keep this document in a safe place.

Explanation of symbols

CAUTION Warnings regarding possible risks of accident or injury.

NOTICE Warnings regarding possible technical damage.

INFORMATION Additional information on the fitting / use.

Please forward the following safety instructions to your patients:

⚠ CAUTION

Use caution near flammables or fire.

Avoid contact with open flames or other sources of heat, as the PE foam cover could catch fire.

The material is hardly inflammable according to ISO 3795. Meets requirements of MVSS 302 standard ≤ 100 mm. **Allowable environmental conditions:** Temperature range for use: -20 °C to +45 °C

NOTICE

Risk of soiling.

Cleaning of the PE foam covers is not possible.

Therefore avoid soiling the PE foam covers.

Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as class I accessories according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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Ottobock HealthCare GmbH