SIGVARIS Armsleeve Donning and Removal

Donning should be done against dry skin (moisturize the evening before).

1) Start by folding from top band down into 1/3 overall length. Gently unfold and move up arm, using folded material as a "handle".

2) Once over the elbow, smooth out any wrinkles and then bend the elbow.

3) Straighten arm and continue pulling the armsleeve up and place the silicon grip top, so it comfortably stays in place on your upper arm.

To remove, gently pull from the grip top inside out and off the arm.

4) **For gauntlet:** To wash hands, do not overstretch seams of thumb. Place other hand inside the gauntlet, pull up and over hand, remove thumb and then fold gauntlet over wrist. Reverse above instructions to put gauntlet back on.

Garment Care

Use a mild detergent. Avoid bleach, softeners or strong detergent.

Hand washing is preferred. Do NOT use dryers. Roll the garment in a towel to remove excess moisture and lay the garment flat or hang to dry. Avoid twisting or overstretching when wet.

It is recommended to replace your armsleeve every 3–6 months. Remeasuring is recommended.

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