Instructions for Use

MALE SINGLE ADAPTER AL
FEMALE SINGLE ADAPTER AL

MALE SINGLE ADAPTER TI
FEMALE SINGLE ADAPTER TI

FEMALE SINGLE ADAPTER JUNIOR
FEMALE SINGLE ADAPTER 5MM OFFSET JUNIOR

EN		Instructions for Use	3
CS		Návod k použití	9
SK		Indikácie použitia	15
RO		Indicații de utilizare	21
HR		Upute za uporabu	27
HU		Használati javallat	33
BG		Указания за употреба	39
SL		Indikácie pre použitie	45

ENGLISH

Medical Device

INTENDED USE

The device is intended as a connection component of a prosthetic system that replaces a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

For more detailed device information, see **Table 1**.

Indications For Use and Target Patient Population

- Lower limb loss, amputation, or deficiency
- No known contraindications

For weight limit and impact level, see **Table 2**.

GENERAL SAFETY INSTRUCTIONS

Warning: Use of a lower limb prosthetic device carries an inherent risk of falling which may lead to injury.

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

Warning: Only use Össur replacement parts for the device for ensured durability.

The device is for single patient use.

ASSEMBLY INSTRUCTIONS

Warning: Ensure tube is fully inserted into Tube Clamp Adapter before tightening.

Only applicable to the Male Single Adapters Short (A-433110, A-435110):

Warning: Do not cut adapter.

Applicable to other devices in **Table 1**:

Cutting the tube:

1. Clamp the tube with a cutting jig or put into a tube cutter.

Caution: Clamping too hard may damage the device.

2. Cut the tube straight and perpendicular.

Warning: Do not cut adapter shorter than the minimum cutting length, see **Table 1**.

3. Remove all burrs and fully insert the tube into adapter.

Additional step for Female Single Adapters:

Warning: For secure fastening use medium strength threadlocker and apply rated torque twice, see **Table 1**.

Warning: When extreme alignment is necessary, use extended set screws to achieve secure fastening.

USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

Environmental Conditions

See which environmental conditions apply to the device in **Table 1**.

A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e. g., rain), no submersion is allowed.

No contact with salt water or chlorinated water is allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

A Waterproof device can be used in a wet or humid environment and submerged in up to 3-meter-deep water for a maximum of 1 hour.

It can tolerate contact with: Salt water, chlorinated water, perspiration, urine, and mild soaps.

It can also tolerate occasional exposure to sand, dust, and dirt.

Continuous exposure is not allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water after exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

MAINTENANCE

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

SPECIFICATIONS

Table 1 - Device Specifications

Part #	Description	Material	Tightening Torque	Environmental Condition	Minimum Cut Length
			Set Screws		
A-433110	Male Single Adapter AL	Aluminum, Stainless Steel	None	Weatherproof	Do not cut
A-433120			None		70 mm
A-442100	Female Single Adapter AL	Aluminum	15 Nm		60 mm
A-442110			15 Nm		60 mm
A-442420			15 Nm		64 mm
A-435110	Male Single Adapter TI	Titanium	None		Waterproof
A-435120			None	70 mm	
A-445100	Female Single Adapter TI		15 Nm	60 mm	
AJ-445100	Female Single Adapter Junior		8 Nm	47 mm	
AJ-445120	Female Single Adapter 5mm Offset Junior		8 Nm	50 mm	
AJ-442100	Female Single Adapter Junior		Aluminum	8 Nm	

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur prosthetic devices are designed and verified to be safe and compatible in combination with each other and custom-made prosthetic sockets with Össur adapters, and when used in accordance with their intended use.

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Compliance

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity, this may correspond to 3-5 years of use.

Table 2 – Usage Specifications

Part #	Impact Level	Weight Limit	ISO 10328 - Label Text
A-433110	The device is for low to moderate impact use, e.g., walking.	100 kg	ISO 10328 – P5 – 100 kg
A-433120			
A-442100			
A-442110			
A-442420			
A-435110	The device is for low to high impact use, e.g., walking and occasional running.	166 kg	ISO 10328 – P7 – 166 kg
A-435120			
A-445100			
AJ-445100	The device is intended for children and adolescent patients.	60 kg	ISO 10328 - P4 - 60 kg
AJ-445120			
AJ-442100		45 kg	ISO 10328 - P3 - 45 kg

ISO 10328 - “P” - “m”kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer’s written instructions on intended use!

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