⚠️ WARNINGS ⚠️

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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Universal Wrist Splint Instructions for Use

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© 2017 Breg, Inc. All rights reserved.

1

Remove dorsal stay and set aside for later application. Apply splint to affected hand and secure with “helping hand” tab.

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2

Apply dorsal stay along the approximate midline. Bend distal end to match bend of wrist.

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3

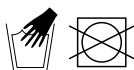
Secure circumferential wrist strap to ensure that the palmer stay is properly seated. Secure remaining closures.

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CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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