Derma Protection ArmComfort

DE	Gebruiksaanweisung (Benutzer)	4
EN	Instructions for Use (User)	6
FR	Instructions d'utilisation (Utilisateur).....	8
IT	Istruzioni per l'uso (Utilizzatore)	10
ES	Instrucciones de uso (Usuario)	12
PT	Manual de utilização (Utilizador)	14
NL	Gebruiksaanwijzing (Gebruiker)	16
SV	Bruksanvisning (Användare).....	18
DA	Brugsanvisning (Bruger).....	20
NO	Bruksanvisning (Bruker)	22
PL	Instrukcja użytkowania (Użytkownik)	24
HU	Használati utasítás (használó)	26
CS	Návod k použití (Uživatel)	28
TR	Kullanma talimatı (Kullanıcı).....	30
EL	Οδηγίες χρήσης (Χρήστης)	32
RU	Руководство по применению (Пользователь)	34

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NOTICE Warnings regarding possible technical damage.

1 Description

Derma Protection ArmComfort improves the suspension of the arm prosthesis while offering increased wearing comfort. Derma Protection ArmComfort protects the upper arm from the upper edge of the socket. It is made of a flexible, knitted polyamide fabric and is lined with a coating of polymer gel.

The polymer gel is extruded with a medical grade mineral oil that is especially gentle to the skin. This mineral oil is released during wearing and, thus, lubricates the skin.

2 Indications

For below elbow amputations.

3 Use and Care

NOTICE

Damage due to incorrect environmental conditions. Do not expose your product to temperatures above 40 °C / 104 °F (e.g. thermal impact caused by hot water, drying on a radiator).

This would destroy or damage the product.

- Roll the Derma Protection ArmComfort inside out to the foam base then roll this section onto your residual limb. (Illus. 1 and 2).
- Roll the bottom end inside out. (Illus. 3)
- Pull on the prosthesis and roll the Derma Arm-Comfort down over the prosthetic socket. (Illus. 4).
- The polymer gel must not be in contact with the cosmetic glove. If necessary shorten the cosmetic glove accordingly.
- Before removing the prosthesis, roll the upper edge of the Derma Protection ArmComfort down and onto the prosthetic socket (Illus. 5 and 6).
When later donning the prosthesis, simply roll it back onto the upper arm.

Derma Protection ArmComfort may be washed as needed in warm water (approx. 30 °C / 86 °F using a mild detergent. Do not wring.

Rinse thoroughly with clear water. Drying time may be longer for the polymer layer.

Drying time may be shortened by drying it with a hand towel. Lay to dry flat either on a rack of towel.

Do not wear your Derma Protection ArmComfort until fully dry!

We recommend that you have two Derma ArmComforts on hand so you will always have one to wear that is clean and dry.

If the suspension of the polymer gel should diminish, lightly powder the inside of your Derma ArmComfort.

For hygienic reasons, and to allow air to reach the skin, we recommend that you not wear the Derma Protection ArmComfort for at least a few hours per day.

INFORMATION

To avoid folds and creases in the inner polymer-gel lining, your Derma Protection ArmComfort should always be stored rolled on its foam bed.

4 Material and Sizes

Flexible, knitted polyamide fabric and polymer-gel-coated lining.

<i>Article no.</i>	<i>For</i>	<i>Size</i>
453A1	Adults	Universal size
453A1=1	Children	Universal size

5 Liability

The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use.

6 CE Conformity

This device meets the requirements of the 93 / 42 / EEC guidelines for medical devices. This device has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Ottobock with sole responsibility according to appendix VII of the guidelines.

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Otto Bock HealthCare GmbH