1E56 Axtion

DE	Gebrauchsanweisung	3
EN	Instructions for use	10
FR	Instructions d'utilisation	18
IT	Istruzioni per l'uso	27
ES	Instrucciones de uso	35
PT	Manual de utilização	43
NL	Gebruiksaanwijzing	51
SV	Bruksanvisning	59
DA	Brugsanvisning	67
NO	Bruksanvisning	74
FI	Käyttöohje	81
PL	Instrukcja użytkowania	89
HU	Használati utasítás	97
CS	Návod k použití	105
RO	Instrucțiuni de utilizare	113
HR	Upute za uporabu	121
SL	Navodila za uporabo	129
SK	Návod na používanie	136
BG	Инструкция за употреба	144
TR	Kullanma talimatı	152
EL	Οδηγίες χρήσης	160
RU	Руководство по применению	169
JA	取扱説明書	178
ZH	使用说明书	185
KO	사용 설명서	192

1

2

INFORMATION

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INFORMATION

Last update: 2017-05-31

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

1.1 Construction and Function

The 1E56 Axtion prosthetic foot is suitable for walking on various surfaces and for a wide range of speeds.

The special combination of carbon and polyurethane absorbs heel strike. The prosthetic foot permits perceptible plantar flexion at heel strike and a natural rollover movement. Stored energy is released by the spring elements.

1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

2 Intended use

2.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

2.2 Area of application

Our components perform optimally when paired with appropriate components based upon weight and mobility grades identifiable by our MOBIS classification information and which have appropriate modular connectors.

The product is recommended for mobility grade 3 (unrestricted outdoor walker) and mobility grade 4 (unrestricted outdoor walker with particularly high demands). Approved for a body weight of **up to 125 kg (275 lbs)**.

2.3 Environmental conditions

Allowable environmental conditions
Temperature range for use: -10 °C to +60°C
Allowable relative humidity 0 % to 90 %, non-condensing

Unallowable environmental conditions
Mechanical vibrations or impacts
Perspiration, urine, fresh water, salt water, acids

Unallowable environmental conditions

Dust, sand, highly hygroscopic particles (e.g. talcum)

2.4 Service life

This product has been tested by the manufacturer with 2 million load cycles according to ISO 22675. Depending on the patient's activity level, this corresponds to a service life of 2 to 3 years.

3 Safety

3.1 Explanation of warning symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

3.2 General safety instructions

Exceeding the service life and reuse on another patient

Risk of injury due to loss of functionality as well as damage to the product

- Ensure that the approved service life is not exceeded.
- Only use the product on a single patient.

Excessive strain on the product

Risk of injury due to breakage of load-bearing components

- Use the product according to the specified area of application (see Page 11).

Unallowable combination of prosthetic components

Risk of injury due to breakage or deformation of the product

- Only combine the product with prosthetic components that are approved for that purpose.
- Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

CAUTION

Using the prosthetic foot without footshell

Risk of injury due to lack of surface grip and loss of functionality of the prosthetic foot

- ▶ Always use the prosthetic foot with a suitable footshell.

NOTICE

Mechanical damage to the product

Change in or loss of functionality due to damage

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

NOTICE

Use under unallowable environmental conditions

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

Signs of changes in or loss of functionality during use

Decreased forefoot resistance or changes in roll-over behaviour are noticeable indications of loss of functionality.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Prosthetic foot	–
1	Protective sock	SL=SPECTRA-SOCK

Quantity	Designation	Reference number
1	Heel wedge kit (soft, hard)	2F20*

Spare parts/accessories (not included in the scope of delivery)	
Designation	Reference number
Foot shell with connection cap	2C5*
Connection cap	2C10*
Changing device	2C100

5 Preparation for use

CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- Observe the alignment and assembly instructions.

CAUTION

Use of talcum

Risk of injury, damage to the product due to lack of lubrication

- Do not use talcum on the product or other prosthesis components.

NOTICE

Grinding the prosthetic foot or footshell

Premature wear resulting from damage to the product

- Do not grind the prosthetic foot or footshell.

5.1 Alignment

INFORMATION

There is a plastic adapter cover on the proximal connection of the prosthetic foot. This protects the connecting section from scratches during the alignment and trial fitting.

- Remove the adapter cover before the patient leaves the workshop/fitting area.

5.1.1 Applying/removing the footshell

> **Recommended tools:** 2C100 changing device

- 1) Slide the prosthetic foot into the footshell.
- 2) Press the heel of the prosthetic foot into the footshell until it engages.

> **Recommended tools:** 2C100 changing device

- 1) Push the lock of the footshell to the rear and pull up the heel of the prosthetic foot.
- 2) Remove the prosthetic foot from the footshell.

5.1.2 Bench Alignment

TT bench alignment

Bench alignment process	
Required materials: 662M4 goniometer, 743S12 heel height measuring apparatus, 743A80 50:50 gauge, alignment apparatus (e.g. 743L200 L.A.S.A.R. Assembly or 743A200 PROS.A. Assembly)	
Perform the assembly and alignment of the prosthetic components in the alignment apparatus according to the following specifications:	
Sagittal plane	
①	Heel height: Effective heel height (shoe heel height – sole thickness of fore-foot) + 5 mm
②	Exterior foot rotation: approx. 5°
③	a–p positioning, middle of the prosthetic foot to the alignment reference line: positioning according to the table below
④	Connect the prosthetic foot and prosthetic socket using the chosen adapters. The instructions for use of the adapters must be observed.
⑤	Determine the centre of the prosthetic socket with the 50:50 gauge. Align the prosthetic socket centrally to the alignment reference line. Socket flexion: Individual residual limb flexion + 5°
Frontal plane	
⑥	Alignment reference line of prosthetic foot: between big toe and second toe Alignment reference line of prosthetic socket: along the lateral patella edge
⑦	Observe the abduction or adduction position.

a–p positioning, middle of the prosthetic foot	
Foot size [cm]	Mid-foot anterior to the alignment reference line [mm]
22 to 25	25 to 30
26 to 28	30
29 to 31	30 to 35

TF bench alignment

- Observe the information in the prosthetic knee joint instructions for use.

5.1.3 Static Alignment

- Ottobock recommends checking the alignment of the prosthesis using the L.A.S.A.R. Posture and adapting it as needed.

- If necessary, the alignment recommendations (TF modular leg prostheses: **646F219***, TT modular leg prostheses: **646F336***) may be requested from Ottobock.

5.1.4 Dynamic Trial Fitting

- Adapt the setup of the prosthesis in the frontal plane and the sagittal plane (e.g. by making angle or slide adjustments) so that correct heel contact and optimal roll-over are ensured.
- **TT fittings:** Make sure that physiological knee movement is achieved after the heel makes impact and the leg begins to bear weight.
- Remove the adapter cover from the prosthetic foot after completion of the dynamic trial fitting and gait training exercises.

5.1.4.1 Optimising the heel characteristics

The heel characteristics are optimised by using heel wedges. In case the heel strike or heel contact is too soft during the mid-stance phase, the heel can be stiffened by means of a heel wedge.

Two heel wedges are available (transparent=soft, black=hard). Ottobock recommends starting with the transparent heel wedge.

- 1) Align the recesses in the heel wedge, in the proximal and posterior directions.
- 2) Press the heel wedge (see fig. 2, item 1) between the top and bottom carbon springs until the heel wedge locks into the recess of the threaded plate (see fig. 2, item 2).
- 3) To remove the heel wedge, push it out to the side.

5.2 Optional: Installing the foam cover

A connecting element (such as a connection plate, connection cap or foam connection cap) provides a detachable connection between the foam cover and the prosthetic foot.

- > **Required materials:** Degreasing cleaner (e.g. 634A58 isopropyl alcohol), 636N9 contact adhesive or 636W17 plastic adhesive
- 1) Measure the distance from the knee rotation point to the desired end of the foam cover and add the compression allowance: For PE foam **10 mm**, for PUR soft foam **30 mm**. For TF prostheses, add twice the compression allowance.
- 2) Cut the pre-shaped foam block to length.
- 3) Pull the foam cover over the prosthesis.
- 4) Set the connecting element onto the footshell or prosthetic foot. Depending on the version, the connecting element engages in the edge or sits on the foot adapter.
- 5) Mount the prosthetic foot on the prosthesis.

- 6) Mark the outer contour of the connecting element on the distal face of the foam cover.
- 7) Dismount the prosthetic foot and remove the connecting element.
- 8) Clean the connecting element using a degreasing cleaner.
- 9) Glue the connecting element onto the distal face of the foam cover according to the marked outer contour.
- 10) Let the glue dry (approx. **10 minutes**).
- 11) Mount the prosthetic foot and adapt the exterior cosmetic cover shape. Take into account compression due to cosmetic stockings or SuperSkin.

6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

7 Maintenance

- ▶ A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

8 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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Otto Bock HealthCare GmbH