

MODEL 117

Bellaire

COMFORT LSO

The **NEW Bellaire Comfort LSO** is a universal lumbar orthosis which provides sagittal control with anterior and posterior rigid panels. It features a flexible pulley system to provide maximum comfort and fit.

- Code **L0627** & **L0642** approved
- One universal size: S-4XL (25-60" circumference)
- Provides superior 5:1 compression and stabilization from L-1 to below L-5 vertebra



Contoured Padded Posterior Panel



Flexible pulley system conforms to the patient and easily adjusts for desired comfort

FEATURES

- Removable rigid posterior panel extends from L-1 to below L-5 vertebra offering equalized compression and support
- Lightweight low profile rigid anterior panel is built into the belt and provides additional abdominal comfort and support
- Easy to use dual handle pulley helps provide equal compression as well as aids patients with arthritic hand issues and muscle weakness

INDICATIONS

- Low back pain, sprains and strains, and scoliosis
- lumbar disc displacement, disc degeneration, disc herniation, and compression fractures
- Post-operative laminectomy, post-operative discectomy, IDET procedure, spondylolisthesis, post-operative fusion, spinal stenosis, and osteoporosis



F 800-521-2192 | 248-588-7480
P 800-923-2537 | 248-588-2960
BeckerOrthopedic.com | f t in


©2020 Becker Orthopedic Appliance Co.
All rights reserved.
Revision 11/30/20

EC REP
Acorn Regulatory Consultancy Services Limited
Knockmorris Cahir Co. Tipperary Ireland, Postcode: E21 R766
P 012 4626 8456
F 012 4626 8648



Fitting Instructions


1 | LOCATE SIZE

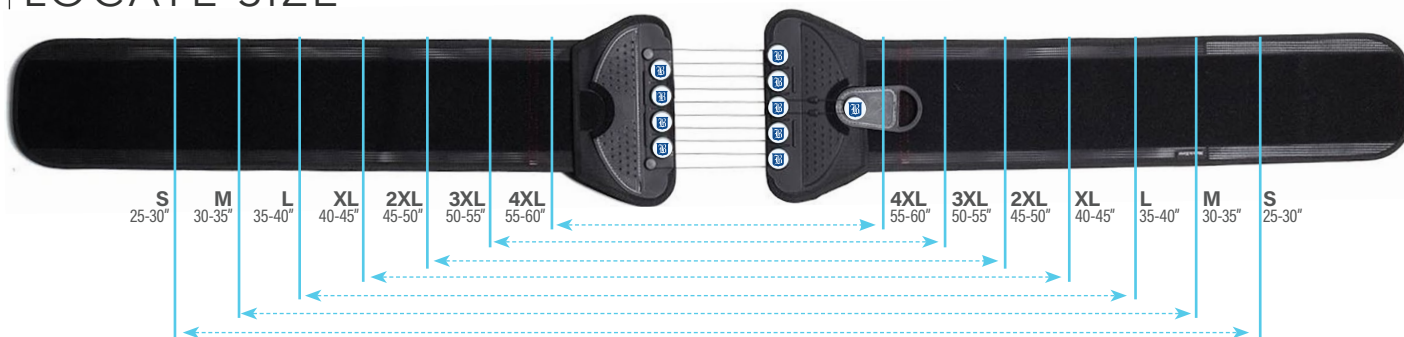

2 | CUT TO SIZE




Lift sleeve and remove extension panels to adjust to appropriate size.

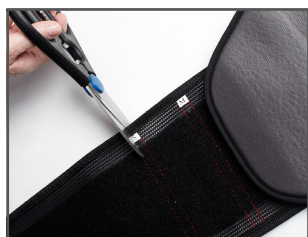

Cut to desired length with scissors (prior to cutting, refer to sizing chart for proper length).




If size adjustment is minimal, simply fold the extension panel to desired length.

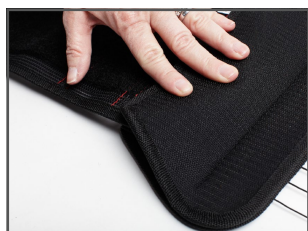

Once adjustment is made, (cut or folded) simply lift the sleeve and attach the extension to Velcro. Press down to secure.

3 | FIT TO SIZE

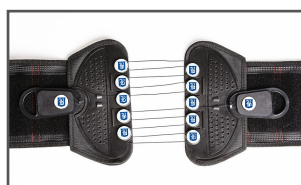

Initially align each pulley handle to the side of the brace prior to fitting. Adjust tension for sitting or standing.



Center back panel on the spine with the pointed part of the posterior panel up.



Wrap the left side to the front, positioning the front panel. Then wrap right side on top of the left panel to close and attach in front.



Pull each side pulley at the same time out and away to desired compression and support.



Place the handles on the fabric for a secure fit.

PULLEY ADJUSTMENT

To Shorten the Length of the Nylon Cord on the Pulley Handle:



Detach the cord from the front clip



Wrap the cord around the 360° side groove clockwise



Reattach the cord to the front clip