

# Bledsoe Flatform Boot

Application Instructions CP020170 Rev E 05/07



Council Directive 93/42/EEC  
of 14 June 1993 concerning  
Medical devices

## FLAT INNERSOLE WALKER FOR POST-OPERATIVE TARSAL & METATARSAL PROCEDURES



**Flat innersole**

This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is generally not intended for patients in excess of 250 lbs. (114 kilos). This device is not intended to prevent injuries or reduce or eliminate the risk of reinjury.

**Indications:** Indicated for post-operative, tarsal, metatarsal, and phalanges procedures requiring a flat innersole surface. Also indicated for metatarsal, forefoot or calcaneus fractures and forefoot sprains.

**Contraindications:** Contraindicated for unstable fractures and proximal tibial or fibular fractures.

**WARNING:** Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. If any additional pain or symptoms occur while using this device, seek medical attention.

**Warranty:** This device is warranted to be free from defects in material and workmanship for a period of 90 days. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at [www.bledsobrace.com](http://www.bledsobrace.com) or by calling (800) 527-3666.

Manufactured by:

**Bledsoe Brace Systems™**

2601 Pinewood, Grand Prairie, Texas, 75051, USA

Toll Free Tel. **1.888.BLED SOE** (253.3763)

Or 1.800.527.3666 • Local Tel. 972.647.0884

Local FAX 972.660.5495

International FAX 972.606.0649

**[www.BledsoeBrace.com](http://www.BledsoeBrace.com)**

European Authorized Representative:



**Emergo Europe**

Molenstraat 15

2513 BH, The Hague

The Netherlands

Tel: (+31) 70 345 8570

Fax: (+31) 70 346 7299

**ORTHOMEDICO** (European Bledsoe Agent)

+32 54 504005 [www.orthomedico.be](http://www.orthomedico.be)

**For product information or questions pertaining to sales or service, please contact your area sales representative or Bledsoe Brace Systems directly.**

# Bledsoe Platform Boot Application Instructions



**1a.** Slip foot into bootie with toes under toe piece. Close forefoot cover over foot. Trim excess leaving 2" (5cm) to 3" (7.6cm) overlap. Close hook to pile closure at front.



**1b.** Wrap calf section around leg, and trim excess leaving 2" (5cm) to 3" (7.6cm) overlap. Close hook to pile closure at front. Remove forefront strap from D-Ring.



**2a.** Slip foot into boot with plastic covers over uprights to prevent engagement of hook to pile closure.



**2b.** Position boot uprights in line with leg and remove plastic covers from uprights to engage the hook to pile closures on uprights with wrap.



**3a.** Place forefoot strap through D-Ring. Place double-sided hook piece in desired position and tighten the strap.



**3b.** After engaging the strap to the double-sided hook piece, trim off excess strap.



**4a.** Starting at the ankle, wrap strap ends snugly forward around sides of leg to interlock strap with uprights.



**4b.** Fasten strap through D-Ring at front of leg and tighten the strap. Repeat this step for remaining two straps.