English

PRODUCT LISTING

SKU	SIZE
PA-627-U	UNIVERSAL

Français

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PROACTIVE WEAR®

Universal 627 LO

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Instruction for Use
Mode d'emploi

800-888-0865 | www.cascade-usa.com

CASCADE ORTHOPEDIC SUPPLY, LP
2638 Aztec Drive | Chico, CA 95928
800-888-0865 | www.cascade-usa.com

PRODUCT DESCRIPTION

The ProActive Wear® Universal 627 LO offers support and compression from L1 to S1. Brace adapts itself to the anatomy and contours to the lordosis of the spine.

INDICATIONS FOR USE

- Post-Operative Stabilization Protocol Following Spina Fusion
- Laminectomy/Laminotomy
- Foraminotomy
- Laparoscopic Disk Replacement
- IDET Procedures
- Multi-Level Decompression

INSTRUCTION FOR PROPER USE

Please follow all prescribing physician's orders for frequency and duration of use.

1. Adjust the size of the belt by opening the lateral pockets and removing the lateral extensions. Cut or fold the lateral extensions to the desired length and resecure into the lateral pockets.
2. Locate the handle on the left side of the device and fasten the handle near the pulley unit.
3. While standing in an upright position, hold the side panels apart, and center the back panel of the device along the spine just above the tailbone.
4. Wrap the left side panel to the front to position the front panel. Then wrap the right side panel on top of the left side panel to close. Attach both sides in front. Adjust the front or back panels as needed.
5. Pull the handle out and away from the pulley unit to reach desired compression and support. Place the handle on the fabric fastener band securely.

MAINTENANCE/CLEANING INSTRUCTIONS

Remove the rigid components. The rigid components can be wiped clean with a damp cloth. Soft goods can be hand-washed in warm water (90 °F—110 °F or 32° C—43 °C) with mild detergent. Triple rinse, squeeze, and air dry. Avoid twisting the material to remove water.

SINGLE PATIENT USE

Single Patient Use Only

WARNINGS AND PRECAUTIONS

Warning: This device does not prevent injury and is not intended to reduce or eliminate the risk of injury.

Warning: Always consult with your physician or healthcare provider before making changes to the device.

Warning: Carefully read all instructions for use before using or fitting this device.

Warning: Proper rehabilitation and activity modification are also a part of a safe treatment program.

Caution: If you experience pain, swelling, or any unusual reaction, contact your physician or healthcare provider immediately.

Caution: This device is intended for single patient use and is not intended to be reused on a second patient.

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