English

PRODUCT LISTING

SKU	SIZE	SIDE
PA-ASA-P	PEDIATRIC	UNIVERSAL
PA-ASA-SM	SMALL	UNIVERSAL
PA-ASA-T	TRAINER	UNIVERSAL
PA-ASA-R	REGULAR	UNIVERSAL

Français

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800-888-0865 | www.cascade-usa.com

PRODUCT DESCRIPTION

The ProActive Wear® Air/Gel Stirrup Ankle provides stability and support to unstable or injured ankles.

INDICATIONS FOR USE

- Support for mild to moderate sprains, strains, post-cast support
- Cold therapy as prescribed by a physician

INSTRUCTION FOR PROPER USE

1. Brace should be worn over sock. Position the rigid plastic shells on each side of the ankle.
2. Adjust bottom hook-and-loop strap if necessary to accommodate width of the foot.
3. Open the Air/Gel Stirrup Ankle Brace and position the heel of the foot evenly on the center pad.
4. Wrap leg straps around the leg and secure with the hook-and-loop fasteners. Readjust if necessary for proper fit and comfort. Readjust the lower and upper straps as needed for a secure and comfortable fit.

MAINTENANCE/CLEANING INSTRUCTIONS

Spot clean with a soft damp cloth and mild soap. Air dry. Do not machine wash or dry.

SINGLE PATIENT USE

Single Patient Use Only

WARNINGS AND PRECAUTIONS

The ProActive Wear® Air/Gel Stirrup Ankle should provide compression but should not be so tight that it restricts circulation. Consult physician or therapist if experiencing pain or swelling.

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