English

PRODUCT LISTING

SKU	SIZE	SIDE
PA-CU-R-XXS	XX-SMALL	RIGHT
PA-CU-R-XS	X-SMALL	RIGHT
PA-CU-R-SM	SMALL	RIGHT
PA-CU-R-MD	MEDIUM	RIGHT
PA-CU-R-LG	LARGE	RIGHT
PA-CU-R-XL	X-LARGE	RIGHT
PA-CU-L-XXS	XX-SMALL	LEFT
PA-CU-L-XS	X-SMALL	LEFT
PA-CU-L-SM	SMALL	LEFT
PA-CU-L-MD	MEDIUM	LEFT
PA-CU-L-LG	LARGE	LEFT
PA-CU-L-XL	X-LARGE	LEFT

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800-888-0865 | www.cascade-usa.com

PRODUCT DESCRIPTION

The ProActive Wear® “Cock-Up” Wrist provides full finger mobility with removable aluminum palmar stay to maintain correct positioning and support.

INDICATIONS FOR USE

- Immobilization of Wrist
- Carpal Tunnel Syndrome

INSTRUCTION FOR PROPER USE

1. Place hand in splint with thumb inside slot. The stay should be resting on the palm and in line with the arm.
2. Wrap and secure closures. For supports with D-rings, feed the straps through D-rings and secure.
3. The supports should be snug but comfortable.

MAINTENANCE/CLEANING INSTRUCTIONS

Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY. **Note:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

SINGLE PATIENT USE

Single Patient Use Only

WARNINGS AND PRECAUTIONS

This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public users. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

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