## English

### PRODUCT LISTING

SKU	SIZE	SIDE
PA-F8-XS	X-SMALL	UNIVERSAL
PA-F8-SM	SMALL	UNIVERSAL
PA-F8-MD	MEDIUM	UNIVERSAL
PA-F8-LG	LARGE	UNIVERSAL
PA-F8-XL	X-LARGE	UNIVERSAL
PA-F8-2X	XX-LARGE	UNIVERSAL

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800-888-0865 | www.cascade-usa.com

**PRODUCT DESCRIPTION**

The ProActive Wear® Figure 8 Ankle Brace provides control and stabilization, aiding in the prevention and recovery of ankle injuries.

**INDICATIONS FOR USE**

- Chronic Ankle Sprains
- Preventative Use

**INSTRUCTION FOR PROPER USE**

1. Before application loosen all straps and laces. Insert the foot inside the boot portion of the brace, making sure to have the tongue of the brace on the top of your foot between your skin and the lacing. While seated and maintaining your ankle at a 90-degree angle, feed lacing through all eyelets and tighten.
2. Bring the inside (medial) stabilizing strap forward across the top of the foot and under the heel. Secure the stabilizing strap to the opposite side of the ankle using the loops provided if necessary. Repeat this process for the outside (lateral) strap.
3. Once both straps are secured ensure optimum performance and support by tensioning both straps at the same time while seated. Use the finger loops and pull up and away firmly on both straps, reapply straps to side of brace.
4. Secure the ankle brace with the elastic closure over the tied laces and secured medial and lateral straps. Apply athletic shoes and wear a pair of socks as you would. Periodic adjustment to the tension of the brace may be necessary depending on your level of activity.

**Note:** Ankle brace should be worn with an athletic sock.

**MAINTENANCE/CLEANING INSTRUCTIONS**

Hand wash in cold water using mild soap. Rinse thoroughly and air dry. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

**SINGLE PATIENT USE**

Single Patient Use Only

**WARNINGS AND PRECAUTIONS**

This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

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