English

PRODUCT LISTING

SKU	HEIGHT	SIZE	SIDE
PA-PHS-WB-S-XS	SHORT	X-SMALL	UNIVERSAL
PA-PHS-WB-S-SM	SHORT	SMALL	UNIVERSAL
PA-PHS-WB-S-MD	SHORT	MEDIUM	UNIVERSAL
PA-PHS-WB-S-LG	SHORT	LARGE	UNIVERSAL
PA-PHS-WB-S-XL	SHORT	X-LARGE	UNIVERSAL
PA-PHS-WB-T-XS	TALL	X-SMALL	UNIVERSAL
PA-PHS-WB-T-SM	TALL	SMALL	UNIVERSAL
PA-PHS-WB-T-MD	TALL	MEDIUM	UNIVERSAL
PA-PHS-WB-T-LG	TALL	LARGE	UNIVERSAL
PA-PHS-WB-T-XL	TALL	X-LARGE	UNIVERSAL

Français

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N° DE RÉF.	HAUTEUR	TAILLE	CÔTÉ
PA-PHS-WB-S-XS	COURTE	T-PETIT	UNIVERSEL
PA-PHS-WB-S-SM	COURTE	PETIT	UNIVERSEL
PA-PHS-WB-S-MD	COURTE	MOYEN	UNIVERSEL
PA-PHS-WB-S-LG	COURTE	GRAND	UNIVERSEL
PA-PHS-WB-S-XL	COURTE	T-GRAND	UNIVERSEL
PA-PHS-WB-T-XS	HAUTE	T-PETIT	UNIVERSEL
PA-PHS-WB-T-SM	HAUTE	PETIT	UNIVERSEL
PA-PHS-WB-T-MD	HAUTE	MOYEN	UNIVERSEL
PA-PHS-WB-T-LG	HAUTE	GRAND	UNIVERSEL
PA-PHS-WB-T-XL	HAUTE	T-GRAND	UNIVERSEL

025826 Rev A

CASCADE ORTHOPEDIC SUPPLY, LP
2638 Aztec Drive | Chico, CA 95928
800-888-0865 | www.cascade-usa.com

PROACTIVE WEAR®

Pneumatic Hard Shell Walking Boot (Short/Tall)

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800-888-0865 | www.cascade-usa.com

PRODUCT DESCRIPTION

The ProActive Wear® Pneumatic Hard-Shell Walking Boot provides a full wraparound shell, offering optimum compression and independent medial and lateral air bladders for optimal control, comfort and fit.

INDICATIONS FOR USE

- Soft Tissue Injuries of The Lower Leg
- Acute Ankle Sprains
- Stress Fractures of The Lower Leg
- Stable Fractures of The Foot and Ankle
- Stable Delayed Union or Non-Union Fractures of Distal Tibia and Fibula
- Achilles Tendon Repairs

INSTRUCTION FOR PROPER USE

1. Unfasten straps, remove the front panel, and open the liner.
2. While seated, place leg in brace, and position heel at the back of the brace. Wrap liner around the leg and foot and secure.
3. Replace the front panel.
4. Fasten the straps from the bottom to the top.
 - For additional control of the ankle, use the alternate location strap.
5. Adjust air bladder compression by pushing the bulbs on either side of the boot. To deflate the air bladders, open the cap of the smaller button next to the bulbs and push the button to release air.

MAINTENANCE/CLEANING INSTRUCTIONS

Hand wash the liner with mild soap and water and lay out to dry. Do NOT machine dry.

SINGLE PATIENT USE

Single Patient Use Only

WARNINGS AND PRECAUTIONS

This product is to be used under the supervision of a medical professional. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

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