## English

### PRODUCT LISTING

SKU	HEIGHT	SIZE	SIDE
PA-P-EU-S-XS	SHORT	X-SMALL	UNIVERSAL
PA-P-EU-S-SM	SHORT	SMALL	UNIVERSAL
PA-P-EU-S-MD	SHORT	MEDIUM	UNIVERSAL
PA-P-EU-S-LG	SHORT	LARGE	UNIVERSAL
PA-P-EU-S-XL	SHORT	X-LARGE	UNIVERSAL
PA-P-EU-T-XS	TALL	X-SMALL	UNIVERSAL
PA-P-EU-T-SM	TALL	SMALL	UNIVERSAL
PA-P-EU-T-MD	TALL	MEDIUM	UNIVERSAL
PA-P-EU-T-LG	TALL	LARGE	UNIVERSAL
PA-P-EU-T-XL	TALL	X-LARGE	UNIVERSAL

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800-888-0865 | www.cascade-usa.com

**PRODUCT DESCRIPTION**

The ProActive Wear® Pneumatic EU Walker protects the foot and ankle, while providing stability and variable compression to patients allowing them to stay mobile during recovery.

**INDICATIONS FOR USE**

- Grade II/III ankle sprain
- Stable fracture of the fibula, talus, calcaneus and medial malleolus
- Post ligament, soft tissue and tendon surgery
- Forefoot or midfoot injury

**INSTRUCTION FOR PROPER USE**

1. Place foot and leg inside foam liner and fasten it snugly.
2. You may put pad kits to supplement inner space or between the walker frame and soft goods.
3. Place the wrapped foot into the Walker. Ensure the malleoli axis is centered to the bilateral upright before removing the plastic covers from the uprights and adhering the foam liner.
4. Fasten all the straps in order and firmly secure them.

**How To Adjust Aircell Pressure**

1. Squeeze the bulb to inflate.
2. To deflate air cells, press the valve which below the bulb.

**How to Set The ROM Hinge**

1. Turn the cover to adjust the stop peg.
2. Remove stop peg to desired degree.
3. Turn the cover back to complete setting.

**MAINTENANCE/CLEANING INSTRUCTIONS**

Washing in cold water, gentle cycle and a mild soap. Do not dry clean or tumble dry. Do not use bleach. Air dry without exposure to any heat source.

**CONTRAINDICATIONS**

- Paraesthesia
- Severe varicose vein
- Skin disorders or injuries in the relevant area
- Impaired lymphatic drainage

**SINGLE PATIENT USE**

Single Patient Use Only

**WARNINGS AND PRECAUTIONS**

Do not wear supports if warming or heat cause allergic reactions please consult your physicians. Do not continuously wear for more than 24 hours.

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