English

PRODUCT LISTING

SKU	LENGTH	SIDE
PA-ROM-12-U	12"	UNIVERSAL
PA-ROM-16-U	16"	UNIVERSAL

Français

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800-888-0865 | www.cascade-usa.com

PRODUCT DESCRIPTION

The ProActive Wear® ROM Wrap Knee Universal is designed for ease of application and provides security while maintaining alignment and ROM of the knee following surgery or injury.

INDICATIONS FOR USE

- Post-Operative
- Post Injury Recovery

INSTRUCTION FOR PROPER USE

1. Open all straps and closures.
2. Align the hinges with the midline of the leg. Ensure that the center of the hinge is inline with the top of the patella (kneecap). Secure the posterior (back) closures of the thigh and calf while maintaining hinge alignment.
3. Secure anterior (front) closures of both thigh and calf. Fasten the posterior (back) straps, followed by the anterior (front) straps.

Note: Straps may be trimmed to length by removing the hook tab, trim the strap to desired length and replace Y hook tab.

MAINTENANCE/CLEANING INSTRUCTIONS

Hand-wash in cold water using mild soap, rinse thoroughly. AIR DRY ONLY. Hinges may be removed for washing. Note: If not rinsed thoroughly residual soap may cause skin irritation and deterioration of the material.

This brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rise with clean, fresh water. AIR DRY ONLY.

SINGLE PATIENT USE

Single Patient Use Only

WARNINGS AND PRECAUTIONS

Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products. If you experience pain or swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

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