

NEW!

EASE

BY THERAFIRM®

EASE ADJUST INELASTIC COMPRESSION BELOW KNEE WRAP

Adjustable gradient inelastic compression wrap
for venous and lymphatic disease.

Compression Level

30–50mmHg*
Firm Support

Designed to provide 30–50mmHg pressure based on wearer needs.


Refer to page 6–7 for more information on compression levels.

How to Measure

1. On the affected leg, measure the circumference of the patient's ankle at the narrowest point, directly above the ankle bone.
2. Measure the circumference of the calf at the widest point.
3. Measure the circumference under the patient's knee, just below the knee bend.
4. Measure the length from the floor to the knee bend (in a straight line).

Use the table below to find the size that corresponds to the measurements.
If any measurement is between sizes, select the larger size.

Size Chart

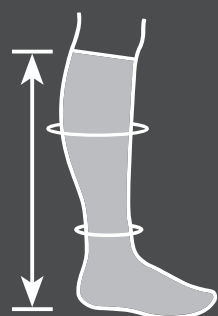

Size	Ankle	Calf	Under Knee
Small	6.5"–9.5" (16–24cm)	9.5"–14" (24–35cm)	10"–14" (25–36cm)
Medium	8"–11" (20–28cm)	12.5"–17" (32–43cm)	12.5"–17" (32–43cm)
Large	9.5"–12.5" (24–32cm)	16"–20" (40–51cm)	15.5"–19.5" (39–50cm)
X-Large	11"–14" (28–36cm)	19"–23" (48–59cm)	18"–22.5" (46–57cm)

Ankle, calf, and length are required for proper fit. Sizes based on average.



Length

Regular

Less than 14"
(36cm)

Tall

More than 14"
(36cm)

**Color
Available**



Black

EASE ADJUST

INELASTIC COMPRESSION BELOW KNEE WRAP

LINER SOCKS INCLUDED

A pair of 10–15mmHg Core-Spun by Therafirm® Socks provide compression and added comfort in the feet and legs

See page 42 for Core-Spun products

DAY OR NIGHT USE

Inelastic materials offer a high working pressure during the day and a low resting pressure with strong containment at night

EASY APPLICATION

Inner sleeve supports the wrap while donning and adjusting straps

MAXIMUM DURABILITY

Made with premium grade materials

PRODUCT GUARANTEE

Therafirm provides a 6-month warranty for materials and workmanship

ANATOMICAL DESIGN

Contours to the calf for a better fit, plus seamless straps reduce pressure points

**NEW!**

30–50mmHg* Ease Adjust Inelastic Compression Below Knee Wrap

Regular Sizes Black

Small Regular	68615
Medium Regular	68625
Large Regular	68635
X-Large Regular	68645

Tall Sizes Black

Small Tall	68715
Medium Tall	68725
Large Tall	68735
X-Large Tall	68745

*The mean compression for an average ankle size.

**Contraindications: Compression products should not be worn and are contraindicated if you have any of the following conditions: arterial insufficiency or degeneration, acute DVT (deep vein thrombosis), untreated congestive heart failure, untreated cancer, untreated infection, absent or severely impaired sensation, allergy to materials, severe or moderate peripheral arterial disease, severe cognitive impairment.