

# Self-care




**easy-slide arm**


easy, safe and simple




**Donning**

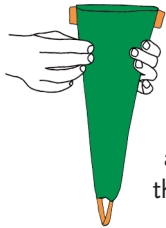

Make sure the product is ready for use by folding back the extra fabric and aligning the top with the product label



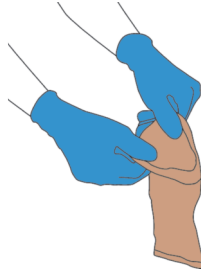


In case of a silicon border, make sure it's folded back before application

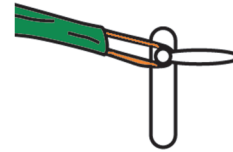

Use a door handle to make it easier pulling out the Easy-Slide Arm

## Donning instructions

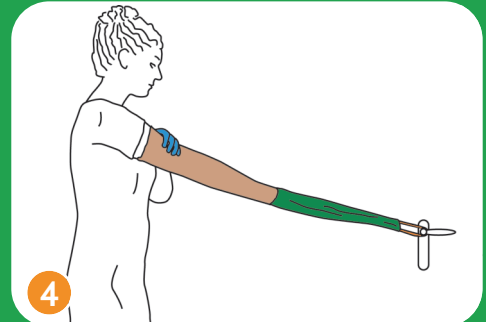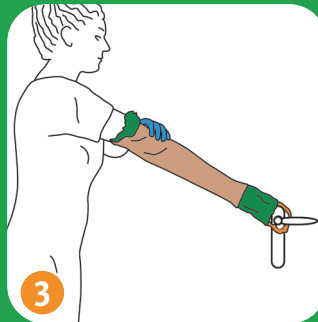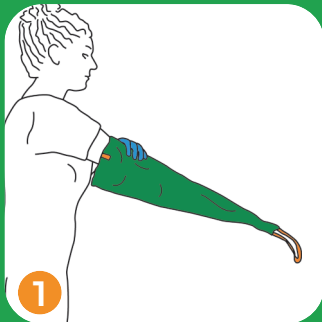

*... We make your life easier!*



# easy-slide arm

easy, safe and simple



Donning



QR code easy-slide Arm

## Application aid for donning **medical compression arm sleeves**

Easy-Slide Arm makes donning compression arm sleeves quick and easy. A straightforward donning method in combination with the application aid requires very little physical effort on the part of the compression wearer or his caregiver. Simply put the aid over your arm, and slide your medical compression sleeve up the limb over the application aid. Easy-Slide Arm helps compression wearers enjoy the convenience of donning compression arm sleeves independently and regularly.

### Easy to Use

- Minimizes friction between compression garment and skin
- Reduces daily stress during donning and doffing
- Increases independence of compression wearers

### Safe to Apply

- Reduces risk of skin damage; can be especially beneficial in case of high skin sensitivity, leg ulcers
- Extends life-span of compression garments
- Helps to avoid garment overstretching or distortion

### Simple Solution

- Offers application aids for all types of compression garments: stockings, sleeves, gloves
- Designed for different shapes and sizes of limbs
- Empowers concordance to compression therapy

### Sizes

Art. Nr.	Size	Length US
E01510	Small	21,5 inch
E01511	Medium	24,5 inch