
2C1, 2C2, 2C3, 2C4, 2C5, 2C6, 2C15


DE Gebrauchsanweisung	3	CS Návod k použití.....	55
EN Instructions for use	7	RO Instrucțiuni de utilizare.....	59
FR Instructions d'utilisation.....	11	HR Upute za uporabu	63
IT Istruzioni per l'uso	15	SL Navodila za uporabo	67
ES Instrucciones de uso	19	SK Návod na používanie	70
PT Manual de utilização.....	23	BG Инструкция за употреба	74
NL Gebruiksaanwijzing.....	27	TR Kullanma talimatı	78
SV Bruksanvisning	31	EL Οδηγίες χρήσης	82
DA Brugsanvisning	35	RU Руководство по применению	86
NO Bruksanvisning	39	JA 取扱説明書.....	91
FI Käyttöohje.....	43	ZH 使用说明书.....	95
PL Instrukcja użytkowania.....	47	KO 사용 설명서.....	98
HU Használati utasítás	51		

1

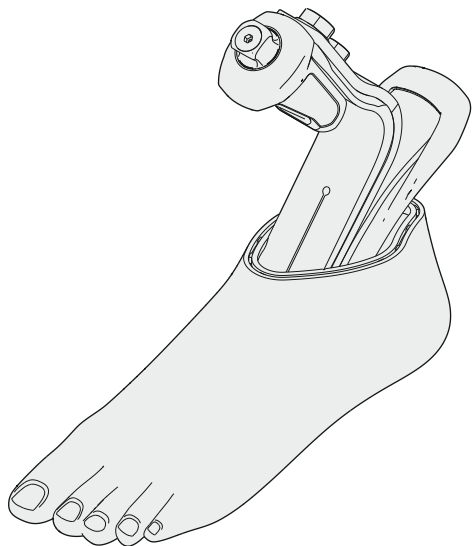2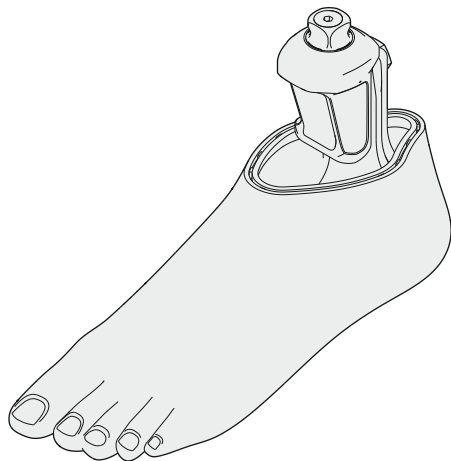

1 Produktbeschreibung

Deutsch

INFORMATION

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INFORMATION

Last update: 2019-09-25

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

1.1 Construction and Function

Footshells are pulled over compatible prosthetic feet. They are required for using the prosthetic feet and give them their natural exterior shape. This document applies for the following footshells: 2C1, 2C2, 2C3, 2C4, 2C5, 2C6, 2C15.

The toenails are **not** paintable.

1.2 Combination possibilities

- Observe the prosthetic foot instructions for use.

Footshell	2C1	2C2	2C3	2C4	2C5	2C6	2C15
Connection	2C19	–	2C10	2C11	2C10	2C19	2C19
cover	2C20		2C20			2C20	2C20

2 Intended use

2.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

2.2 Environmental conditions

Allowable environmental conditions

Temperature range for use: -10 °C to +60°C

Allowable relative humidity 0 % to 90 %, non-condensing

Unallowable environmental conditions

Mechanical vibrations or impacts

Perspiration, urine, fresh water, salt water, acids

Dust, sand, highly hygroscopic particles (e.g. talcum)

Other environmental conditions

UV light

Not resistant

Hydrolysis

Not resistant

2.3 Service life

The product is a wear part, which means it is subject to normal wear and tear.

3 Safety

3.1 Explanation of warning symbols

CAUTION

Warning regarding possible risks of accident or injury.

NOTICE

Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION

Unallowable combination of prosthetic components

Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved for that purpose.
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

CAUTION

Use of a worn product and reuse on another patient

Risk of injury due to loss of functionality as well as damage to the product

- ▶ Inspect the product regularly for wear. Do not continue using a worn product.
- ▶ Only use the product for a single patient.

CAUTION

Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

NOTICE

Use under unallowable environmental conditions

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

NOTICE

Contact with salt water or water containing chlorine as well as abrasive substances (e.g. sand)

Damage and premature product wear and tear

- ▶ Promptly clean the product following any contact with the substances identified above.

NOTICE

Mechanical damage to the product

Change in or loss of functionality due to damage

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern and by the development of noises.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Footshell	–
1	Connection cover	–

5 Preparation for use

CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- Observe the alignment and assembly instructions.

NOTICE

Grinding the footshell

Premature wear resulting from damage to the footshell

- Do not grind the footshell.
- **Inform the patient.**

5.1 Applying/removing the footshell

Putting on the footshell

> **Recommended tools:** 2C100 changing device

- 1) Slide the prosthetic foot into the footshell.
- 2) Press the heel of the prosthetic foot into the footshell until it engages.

Removing the footshell

> **Recommended tools:** 2C100 changing device

- 1) Push the lock of the footshell to the rear and pull up the heel of the prosthetic foot.
- 2) Remove the prosthetic foot from the footshell.

6 Use

CAUTION

Use of the product without shoes

Risk of injury due to slipping on slippery surfaces

- Do not use the product on slippery surfaces.

6.1 Cleaning

CAUTION

Use of unsuitable cleaning agents or disinfectants

Impairment of functionality and damage due to incorrect cleaning agents or disinfectants

- Only clean the product with the approved cleaning agents.
- Only disinfect the product with the approved disinfectants.
- Observe the instructions for cleaning and care.

> **Recommended cleaning agent:** pH neutral soap (e.g. 453H10 Derma Clean)

- 1) Clean the product with clear fresh water and a pH neutral soap.
- 2) Rinse the soap away with clear fresh water.
- 3) Dry the product with a soft cloth.
- 4) Allow to air dry in order to remove residual moisture.

7 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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