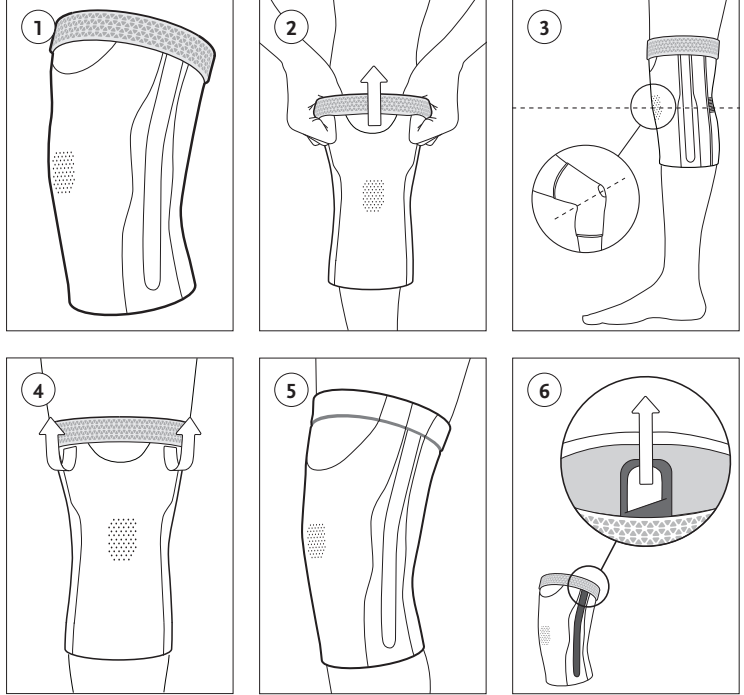
ENGLISH

INDICATIONS FOR USE

For knee conditions that may benefit from compression, such as:

- Mild sprains and strains
- Knee pain
- Feeling of instability

CONTRAINDICATION

If you have a medical condition that decreases blood circulation in your limbs, then consult your doctor before you use the brace.

Such medical conditions include, but are not limited to, diabetes or peripheral vascular disease.

WARNINGS

- Contains natural rubber latex which can cause allergic reactions.
- Do not use for extended periods of time. Remove before you go to bed, or rest for extended periods of time.
- Immediately remove and stop use of the brace and consult your doctor if you notice:
 - pain
 - swelling
 - changes in skin sensation
 - blood flow is not sufficient
 - extremities turn blue, white or cold
- This product is designed and tested for single patient usage. Do not use for multiple patients.
- This product is designed to be worn directly on the skin.
- Do not wear on an open wound.
- Examine your knee and the brace after each use.

PRODUCT MATERIAL LIST

Rubber, polyester, polyamide, elastane, cotton, polyurethane, steel.

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CARE INSTRUCTIONS

- Wash on a gentle cycle at max 30°C/ 86°F with mild detergent and air dry. Do not tumble dry.
- Wash regularly to keep the compression effect.
- The steel stays can be removed if necessary.

FITTING INSTRUCTIONS

NOTE:
If between sizes on the sizing chart, always select the smaller size for optimal fit.

Putting the brace on

1. Fold the sleeve down above the colored ring at the top, until the grip pattern is fully shown (**Figure 1**).
2. Hold the sleeve and pull it up (**Figure 2**).
3. Position the knee cap in the middle of the dotted knee cover (**Figure 3**).
4. Fold the sleeve up (**Figure 4**).
5. The brace is now correctly on the knee (**Figure 5**).

Removing the steel stays

1. Fold the sleeve down above the colored ring at the top end, until the grip pattern is fully shown (**Figure 1**).
2. Then pull out the steel stays from inside the sleeve (**Figure 6**).

DISPOSAL

All components of the product and packaging should be disposed of in accordance with respective national environmental regulations. Users should contact their local governmental office for information on how these items can be disposed of in an environmentally sound manner.

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