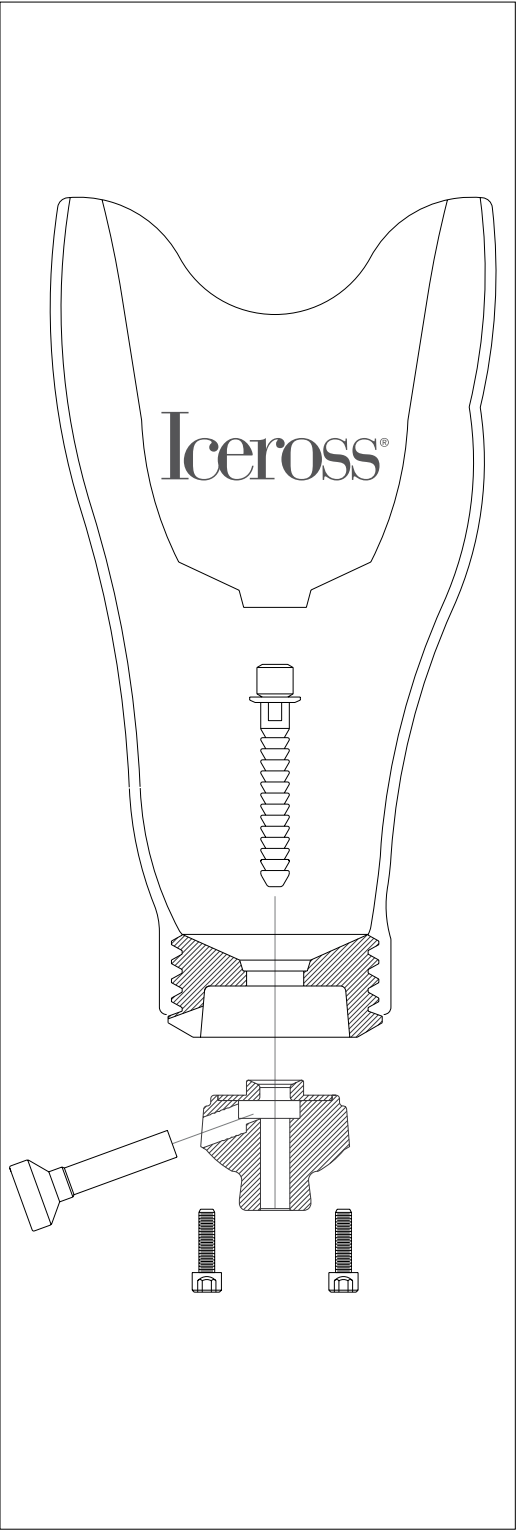
ENGLISH

ICELOCK® RATCHET 100 SERIES L-125000,

ICELOCK® RATCHET SP 125

Align the Lock Body (1) in the Puck (2). Apply Loctite 425 on the four screws (3) and torque them to 10Nm (7.3ft.lb), using a 4mm hexagon bit.

RELEASE UNIT

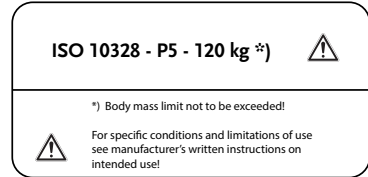
Shorten the release tube and pin (5) equally to the required length. Insert the pin into the tube and screw the tube firmly and completely into the hole.

ATTACHMENT PIN

Apply Loctite 425 onto the threads of the Attachment Pin (6) and screw it to the ICEROSS. Torque the Attachment Pin to 2Nm (1.5ft.lb) with a 7mm wrench.

Note: Use the Attachment Pins for ICELOCK 100 series only.

- 1 year warranty
- Rated up to high activity level (K3)
- 120kg/265lbs

Important: For best results, ensure that the pin is fully engaged. If temporary swelling prevents full engagement, a minimum of four teeth must enter the lock for safety reasons.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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