ENGLISH

CONTENT

L-292011 Lock Body 4-Hole with Guide (w/o Clutch Mechanism).
L-292020 Clutch Mechanism and T-wrench (L-293060 T-wrench).
L-292000 Attachment Pin clutch, standard.
L-292125 Lam/Therm Ring.
L-292118 Delrin Tooling Body.
L-292129 Delrin Screw.
L-293041 Screw M6x20.
L-293042 Screw M6x35.

FOR LAMINATION

1. Flatten distal end of Plaster Cast (3) enough to centrally attach Delrin Tooling Body (2). Attach with nails provided or double-sided tape maintaining original pin alignment.
2. Apply inner PVA bag. Pull through central hole of Lam/Therm Ring (4). Flare bag away from central hole and attach ring using Delrin Screw (1), sealing PVA bag in the process.

NOTE: Ensure that arrow (A) on Lam/- Therm Ring (4) is aligned in AP or ML direction.

3. Place some clay in spacing screw heads (B) and in groove on Delrin Screw (C).
4. Tie in lay-up material. Tie all but first layer beyond Lam/Therm Ring. Ensure that four spacing (B) screws are embedded evenly in material.
5. Apply outer PVA bag and complete lamination.
6. Grind distal end accurately to expose spacer screw heads (B). **DO NOT grind any deeper!**
7. Remove spacer screws (B) and attach Lock Body (5) with four long countersunk screws (6). Use Loctite 410 or 411 and torque to 12 Nm (9 ft.lbs.).

NOTE: Push Button (8) may be shortened by unscrewing from lock body and cutting it with hacksaw.

FOR THERMOPLASTICS

1. Flatten distal end of Plaster Cast (3) enough to centrally attach Delrin Tooling Body (2). Attach with nails provided or double-sided tape maintaining original pin alignment.
2. A small hole may be drilled from edge of Delrin Tooling Body (D) through length of Plaster Cast (3) to assure maximum vacuum.
3. Attach Lam/Therm Ring (4) to Delrin Tooling Body (2) with Delrin Screw (1).

NOTE: Push Button (8) may be shortened by unscrewing from lock body and cutting it with hacksaw.

4. Vacuum form thermoplastic. Ensure proper forming, especially at distal end. It is important that thermoplastic is at least 6 mm thick at distal end.
5. Rough-cut and grind plastic in transverse plane, stopping at spacer screws (B). **DO NOT grind any deeper!**
6. Remove spacer screws (B) and attach Lock Body (5) with four long countersunk screws (6). Use Loctite 410 or 411 and torque to 12 Nm (9 ft.lbs.).

NOTE: Push Button (8) may be shortened by cutting it with hacksaw .
IMPORTANT: If Clutch Mechanism (7) is removed during lifetime of lock, torque to 15Nm (11 ft./lb).

ATTACHMENT PIN

Apply Loctite 410 or 411 onto threads (E) on Attachment Pin (10) and screw it into Locking Liner. Torque Attachment Pin to 4Nm (3ft.lb). Also available:


- L-292003 Attachment Pin clutch, short (7 notches).
- L-292005 Attachment Pin clutch, long (12 notches).

NOTE: Use attachment pins for Icelock 200 series only.

T-wrench (9) supplied is specially designed to wind Push Button (8) on Clutch Mechanism (7) when donning socket. Use of any other tool may result in over tightening.

- Rated up to 120kg (265lbs) - high activity level (K3).
- Limited one-year warranty.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water spraying against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

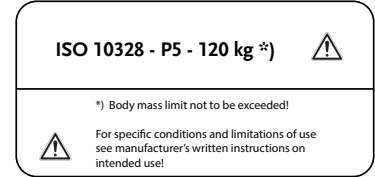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EN – Caution: Ossur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards is achieved only when Ossur products are used with other recommended Ossur components. If unusual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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