Iceross Seal In® X5 Liner

For Transfemoral users

Life Without Limitations®

.....	3
(EN) Instructions for use.....	5
(DE) Gebrauchsanweisung.....	10
(FR) Guide de fabrication.....	15
(ES) Instrucciones para el uso.....	20
(IT) Istruzioni per l'uso.....	25
(EL) Οδηγίες χρήσης.....	31

- **Caution:** Ossur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Ossur products and components are used with other recommended Ossur or authorized components. If un-usual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist.

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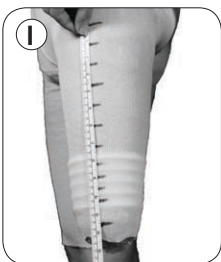
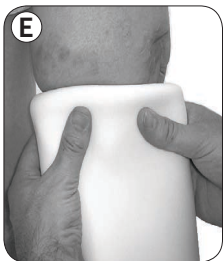
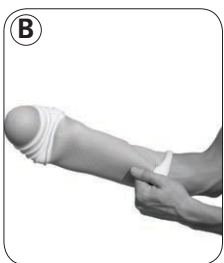
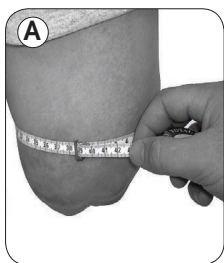
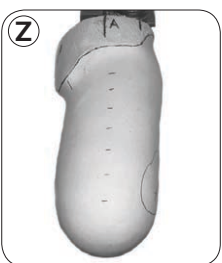
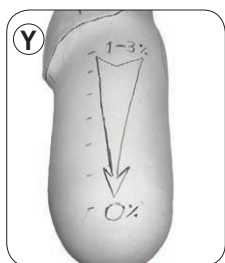
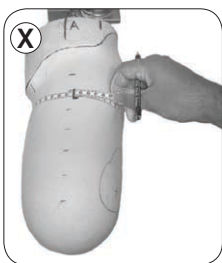
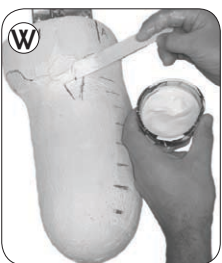
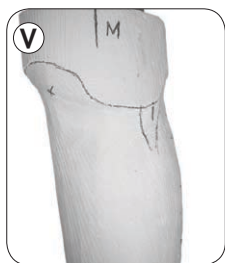
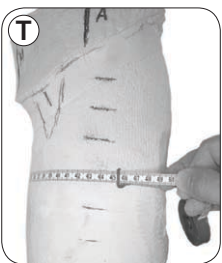
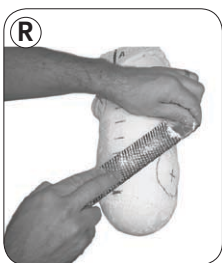
ENGLISH

INTRODUCTION

Iceross Seal-In® X5 Liner is a unique Iceross Liner, which incorporates 5 serial Seals along the length of the liner.

The system offers firm vacuum suspension, protection of soft tissues and comfort to the Transfemoral user.

The Iceross Seal-In® X5 liner is available in standard and conical profile with specially formulated silicone to help stabilise soft tissue. The liner incorporates Active Skin Care and a full length matrix both required for user comfort and optimal performance.

The successful fitting of the transfemoral prosthesis is dependent on the fit of the hard socket. Appropriate socket design is fundamental for comfort, control and suspension to be achieved. The following hand casting instructions are intended as a recommendation to help create a well fitting socket accommodating the Iceross Seal-In® X5 Liner for transfemoral users.

It has been shown that the Iceross Seal-In® X5 Liner performs best in conjunction with an ischial containment brim shape, however good results have been achieved with many different socket designs. The Iceross Seal-In® X5 Liner is recommended for use with both thermoplastic and acrylic lamination socket designs. For short residual limbs a special consideration should be taken as loss of suspension may be experienced. (see **CONTRAINDICATIONS FOR USE**)

Please read the following hand casting and rectification instructions carefully before starting the procedure.

EQUIPMENT

- Plaster of Paris Bandages 15cm and 20cm
- Plastic Film
- Casting Pants (non-elastic)
- Indelible Pencil
- Tape Measure
- Clamps x2
- Scissors

PATIENT ASSESSMENT

Carry out full assessment of the amputee taking care to assess the following:

- Hand strength and dexterity is sufficient to roll the liner onto their residual limb.
- A degree of user cognitive ability is required to correctly and safely use the liner on a daily basis. Inspect residual limb. Knowledge of anatomical landmarks, scarring and sensitive areas will aid in socket fitting.
- Residual Limb is of sufficient length to allow up to 3 Seals to be housed fully within the socket. (see **CONTRAINDICATIONS FOR USE**)
- The residual limb needs to be stable, with less than 3 ply sock daily fluctuation. Collect relevant prosthetic history and standard measurements.

CONTRAINDICATIONS FOR USE

Minimum recommended length:

Measure the residual limb length with the tissue hanging down, from the perineum to the distal end.

- Compare the measurement to the table below.
- Low trim lines of the hard socket can affect suspension and should be assessed on individual basis.

Liner Sizes	Minimum Length
18 – 23	11,0 cm
25 – 30	12,5 cm
32 – 40	13,5 cm
45 – 50	15,5 cm
55	16,5 cm

ICEROSS SEAL-IN® X5 LINER SELECTION

LINER SIZING :

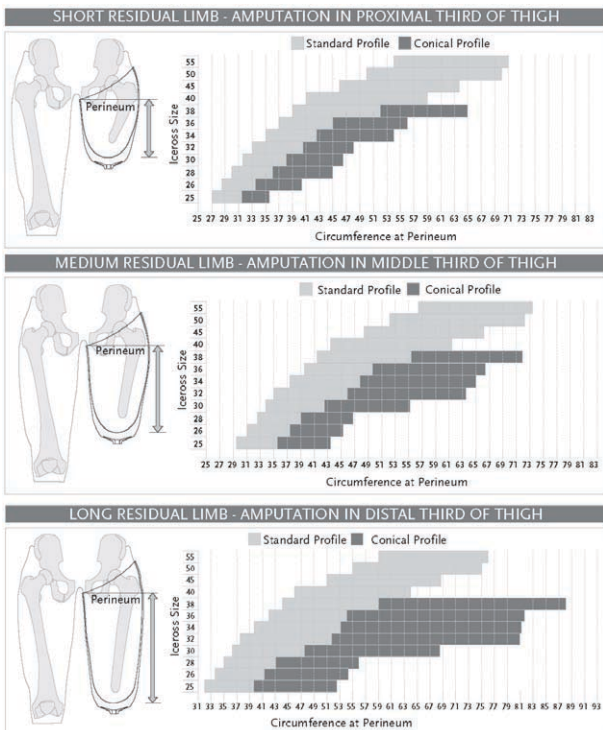
To determine the correct Iceross size:

1. Measure the circumference of the residual limb at 4cm from the distal end with the tissue hanging down. **(Figure A)**
2. Choose the Iceross size, which corresponds to the measurement or which, is immediately below the indicated measurement.
3. **DO NOT** choose a liner size, which exceeds the measurement, or round up.

EXAMPLE: If the residual limb measures 31cm at the indicated location, choose size 30 **NOT** size 32.

PROFILE:

Measure and record the circumference of the residual limb at the level of the perineum, refer to the horizontal axes

Selecting the correct size is very important.

If the liner is too tight:

- Pistoning and/or numbness and blistering may occur.
- The seals can produce excessive ring tension to the residual limb.

If the liner is too loose:

- Increased perspiration and/or movement of the limb inside the liner may occur.
- Perspiration and movement can lead to blisters and rashes.

DONNING TECHNIQUE

Before donning a new liner clean it with lukewarm water, and dry it with lint free cloth as the natural oils in the silicone make it difficult to grip.

1. Turn the liner inside out by gripping the distal end from the inside as shown in the illustration. **(Figure B)**
2. Grip the top of the liner and slide it over the hand until the inner surface is fully exposed. **(Figure C)**
3. Spray the Clean & Simple Lubricant Spray into the inverted liner for easier roll on. **(Figure D)**
4. After exposing as much of the distal end of the liner as possible, position it against the residual limb and with light compression roll upward onto the limb. Check that no air pockets are present. **(Figure E)**
5. Roll the liner all the way up the limb, taking care not to damage it with fingernails. Do not tug or pull.
6. Ensure the Seals sit horizontally around their full circumference.

NOTE: Be very careful that the inside of the liner is clean, dry and free from any foreign objects that can cause skin irritation.

NOTE: With use the silicone will soften, which will facilitate application. Ensure no air pockets are trapped during rolling as this could lead to skin complaints or excessive perspiration.

Determine Iceross trim-line.

Mark and trim above intended trim-line of hard socket. **(Figure F)**

Leave Iceross Seal-In® X5 on for a short time ensuring there is no discomfort

NOTE: It is recommended that the Iceross Seal-In® X5 Liner is trimmed with a Curvemaster. However sharp scissors may be used for the initial cut.

PREPERATION FOR CASTING

1. Apply single layer of plastic wrap over ICEROSS Seal-In® X5 Liner. Ensure all areas are covered. **(Figure G)**
2. Wrap a single layer of plastic wrap around waist.
3. Apply a pair of casting pants; use shoulder straps to suspend if necessary. **(Figure H)**

IDENTIFY AND MARK THE FOLLOWING

- Adductor longus tendon
- Anterior superior iliac spine **(Figure I)**
- Greater trochanter
- Ischial tuberosity
- End of femur on lateral side. Mark proximal trimline **(Figure J)**
- Anterior, inguinal crease, allowing clearance for anterior superior iliac spine during full flexion
- Lateral, superior to greater trochanter
- Posterior, containing gluteal muscles
- Medial, containing ischial tuberosity

Suggested measurements:

- Length from perineum to distal end of ICEROSS
- Circumference measures at 3cm intervals down residual limb. (Important – note) **(Figure I)**
- Ischial tuberosity to adductor longus tendon
- Medial border of ischial tuberosity to greater trochanter (skeletal medial/lateral)
- Angle of adduction and flexion of the limb
- Skeletal M/L
- Soft tissue M/L

CASTING

Plaster of Paris (POP) preparation:

- 1 Prepare the lateral slab, a 15 cm, 12 ply slab of POP bandage long

enough to reach from the proximal trimline to the distal end of the residual limb. **(Figure K)**

2. Prepare the proximal slab, a 20cm, 4-ply slab of POP bandage long enough to wrap fully around the waist.
3. Prepare the distal slab, a 15cm, 6ply slab to close the distal end.
4. Prepare the perineal slab, a 15 cm, 6 ply slab of POP bandage long enough to reach from the full circumference of the proximal trimlines.

Plaster of Paris (POP) application

1. Apply the lateral slab to the residual limb and form firmly. **(Figure L)**
2. Wrap the proximal slab around the thigh and waist. Reinforce proximal slab by wrapping with a 15cm bandage; continue around contralateral hips to aid lateral shaping. **(Figure M)**
3. Wrap remainder of the residual limb with a 15cm bandage.
4. Close the distal end with a 6 ply slab of plaster Paris bandage. **(Figure N)**
5. Position residual limb in abduction.
 - a. Apply the perineal slab, locate middle against perineum. **(Figure O)**
 - b. Anterior: Cover inguinal fold cross above greater trochanter.
 - c. Posterior: Cover ischial tuberosity well, follow proximal to gluteal crease cross above greater trochanter.
 - d. Position residual limb in slight flexion and adduction.
 - e. Ensure pelvis is level.
6. Locate hands to reinforce proximal shape. **(Figure P)**
 - a. **Hand 1:** Use web between thumb and forefinger to relieve adductor longus tendon. Reinforce ischial control area with fingers.
 - b. **Hand 2:** Mould lateral wall around greater trochanter and lateral femur.
Note: If ischial tuberosity can not be reached, position hand more vertically and use fingertips to reinforce ischial control area.
7. Once plaster has fully set carefully remove negative cast, and cut shorts to help removal. **(Figure Q)**
 - a. Mark alignment reference lines for socket set-up.
 - b. Fill the cast with Plaster of Paris in preparation for modification.

MODIFICATION

1. Remove any ridge present over the series of Seals. **(Figure R)**
2. Transfer alignment reference lines to the positive. **(Figure S)**
 - a. Ensure all landmarks transfer to positive.
 - b. Mark the AP/ML orientation lines.
3. Check length and circumference measures. Check AP and ML 4. measures. **(Figure T)**
4. Define anterior trimline. **(Figure U)**
5. Define lateral trimline. **(Figure V)**
 - a. Reduce cast above and posterior to trochanter by 1cm.
 - b. Define posterior/medial trim line.
 - c. Define the medial trim-line.
6. Relieve adductor longus tendon and pubic ramus. **(Figure W)**
7. Check circumference measure. **(Figure X)**
8. Calculate volume reduction. Recommended 1-3% proximally, reducing to 0% distally. **(Figure Y)**

NOTE: Volume is one of the most important features of the socket. Ensure all measurements are accurate. Reduce the model to the calculated reductions, material may be removed from the following areas:

- Posterior/lateral corner
- Anterior femoral triangle (scarpas triangle)
- Along lateral femoral shaft avoiding distal femur

- Below ischial tuberosity

NOTE: It is very important that cast surface is smooth as it may affect the seal of the Seals. (**Figure Z**)

CHECK SOCKET FITTING

It is advisable to assess the fit with a check socket prior to fabrication of the final socket.

The Iceross Seal-In® X5 Liner is recommended for use with the Icelock® Expulsion Valve 551 (product # L-551000) or 552 (product # L-552000). Refer to the Icelock Expulsion Valve Instructions For Use for installation procedure. (**Figure A1**)

DONNING THE PROSTHESIS

1. Lightly spray the Seals around their full circumference, and inside the hard socket for smoother entry.
2. Align the prosthesis as instructed by the prosthetist and push into the hard socket expelling air.

NOTE: If the user is having difficulty donning the prosthesis, more lubricant may be needed or the valve is blocked.

NOTE: If the user feels the skin of the residual limb is being stretched to one side after donning the prosthesis, the distribution of the lubricant is inadequate around the full circumference of the Seals.

CHECKPOINTS

Volume:

- Maximum contact of Seals around the circumference of the socket
- No air gaps under load. Proximal Shape
- Intimate cosmetic fitting around the trimline
- Full flexion possible without interference on iliac spine
- Adduction not impaired by medial brim
- Ischium contained comfortably and functioning to stabilize the socket against lateral movement
- Clearance for pubic ramus and adductor longus tendon Suspension
- Little or no pistoning
- No visible movement with sustained pull
- Rotational stability
- Comfortable forced abduction
- Comfortable flexion and extension
- Full flexion possible whilst maintaining good suspension

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Iceross Seal In X5[®] is covered by one or more of the following patents. Other U.S. and foreign patents pending. [®] indicates trademark registration in U.S.A. and selected countries only.