





Kinetic Light with Pylon^{K2}

- K2 foot with integrated multi-axial ankle
- Proximal male pyramid
- Full length, composite keel
- Sandal toe foot shell (non-removable)
- Integrated ankle function for all categories is the same as the existing Kinetic

Indications: Low to medium impact level, K2

Selection

Model No.	Description	Size	Weight Limit
SKF100	 Kinetic Light, Light Foot Shell	22-30 cm	160 kg / 350 lb*
SKF103	 Kinetic Light, Dark Foot Shell	22-30 cm	160 kg / 350 lb*

Bumper Selection Instructions

To optimize selection, follow the three steps below to determine the appropriate category.

1. Locate the column associated with the amputee's foot length.
2. Locate the row corresponding to the amputee's weight.
3. The area where the column and row intersect lists the appropriate category.

Important Note: If the amputee has a long BK or carries heavy loads choose the next category higher.

Warning: Choosing a lower category than what is suggested based on the above procedure and patient data will void the warranty. If your patient's weight exceeds the limits of the chart, please contact Trulife Customer Service.

Foot Length	22	23	24	25	26	27	28	29	30
Weight									
137-160 kg 301-350 lb	4	4	5	5	5	5	5	5	5
101-136 kg 221-300 lb	3	3	4	4	4	4	4	4	4
81-100 kg 177-220 lb	2	2	3	3	3	3	3	3	3
61-80 kg 133-176 lb	1	1	2	2	2	2	2	2	2
< 60 kg < 132 lb	1	1	1	1	1	1	1	1	1

Specifications

Weight Limit	160 kg / 350 lb*
Product Sizes	22-30 cm
Toe	Sandal Toe
Foot Shell	Light, Dark
Product Weight Cat 1-2: Cat 3-5:	481 g / 1.06 lb 564 g / 1.24 lb (Size 25 without Pylon)
Min. Clearance	7.3 cm / 2.9 in
Heel Rise	9.5 mm / 3.8 in
Warranty Foot: Foot Shell:	2 years 1 year
Includes	Female Pylon (AAA200)
Made in USA	

*For category #5



AAA225
Clamp Adapter
(page 26)

AAA100
Adjustable Height
Clamp Adapter
(page 26)