medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita, n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
 T: +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.meditandmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

medi Polska Sp. z o.o.
 Zygmunt Starego 26
 44-100 Gliwice
 Poland
 T +48-32 230 60 21
 F +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351-21 843 71 60
 F +351-21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
 T +7-495 374 04 56
 F +7-495 374 04 56
 info@medirus.ru
 www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
 Rm 1806, Building 1 Sandhill Plaza
 No. 2290 Zuchongzhi Road
 Pudong New District
 200131 Shanghai
 People's Republic of China
 T: +86-21 61761988
 F: +86-21 61769607
 infocn@medi.cn

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS
 Kültür mah. çaldıran sok.14/3
 06420 Çankaya/Ankara
 Turkey
 T +90 312 435 20 26
 F +90 312 434 22 82
 info@medi-turk.com
 www.medi-turk.com

medi UK Ltd.
 Plough Lane
 Hereford HR4 OEL
 Great Britain
 T +44-1432 37 35 00
 F +44-1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

medi Ukraine LLC
 Tankova Str. 8, office 35
 Business-center „Flora Park“
 Kiev 04112
 Ukraine
 T +380 44 591 11 63
 F +380 44 392 73 73
 info@medi.ua
 http://medi.ua

medi USA L.P.
 6481 Franz Warner Parkway
 Whitsett, N.C. 27377-3000
 USA
 T +1-336 4 49 44 40
 F +1-888 5 70 45 54
 info@mediusa.com
 www.mediusa.com

medi

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Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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M.4s OA comfort

Intended purpose

M.4s OA comfort is a varus/valgus brace. The product is exclusively to be used for the orthotic fitting of the knee and only on unbroken skin or if any wounds have been properly covered.

Indications

- Severe medial or lateral compartment osteoarthritis (class III at least) in conjunction with complex ligament instability.
- Unicompartmental load relief before HTO (brace test)
- To relieve the load for painful stress fractures (e.g. tibial plateau)
- Cartilage surgery / mosaicplasty / meniscus repair / corrective osteotomy which require(s) post-operative load relief of the medial or lateral compartment
- Extreme overload of the medial and lateral compartments

Contra-indications

Bilateral (bicondylar) osteoarthritis of the knee

Putting on the brace

- Open all straps and pull the numbered ends out of the slots on the belt. Tip: To stop the straps from sticking to each other, you should fasten the velcro part after this step.
- Sit on the edge of a chair and pull the brace over the affected leg. Bend your leg at approximately 70°.
- Position the brace so that the centre of the joint (middle of the extension wedges) sits level with the top edge of the kneecap and slightly behind the leg's lateral central line. Please make

sure that the brace has been placed on the leg straight and is not twisted.

- Fasten strap 1 first and then strap 2.
- Keep your leg bent and check again that the brace is positioned correctly.
- Then, fasten straps 3 to 6 in order.
- Take a few steps in order to ensure that the brace is sitting correctly. If required, the straps can be adjusted, in the same order, to ensure that the brace is fitted correctly.
- To take off the brace, open all of the straps and pull the brace off.

Adjusting for Varus/Valgus Load Relief

Have your technician explain the adjustment options for the M.4s OA to you so that you are clear on them. This will enable you to find a way to manipulate the brace safely so that you can benefit from optimal support from the brace as quickly as possible.

Setting Load Relief: for Professionals

Set the load relief mechanism to neutral. This is achieved when the pressure component lies flat against the side of the knee joint and the patient can only feel slight pressure. Ensure that both brace joints are still parallel to each other. Then increase the pressure by turning all of the setting screws a quarter-turn towards "+". This is a good base setting for the first wearing period. Let the patient walk around wearing the brace for a few minutes. If necessary, adjust the setting screws again.

Setting the restrictions for flexion and extension (can only be carried out by an orthopaedic technician)

Extension restriction wedges, silver (stretch restriction): 0°, 10°, 20°, 30°, 45°
Flexion restriction wedges, black (bend restriction):
0°, 10°, 20°, 30°, 45°, 60°, 75°, 90°
Can be immobilised at:
0°, 10°, 20°, 30°, 45°

The medial (internal) and lateral (external) degree settings must always be the same.

The brace is fitted with a 0° extension wedge. Other wedges are included with the brace.

Select the required wedge (extension wedges are marked with “Ext”, flexion wedges are marked with “Flex”, the relevant number of degrees is also stated).

Place the wedge in the brace joint and lock it with the screw.

Move the brace joints and check that the wedges are securely in place and are well-positioned.

Important Note:

Only change the extension and flexion restrictions under the orders of the doctor treating you. To avoid hyperextension, extension wedges or 0° wedges must always be used. The wedges set the exact radius of movement for the brace. If soft tissue conditions pose issues, then it may be necessary to use a larger wedge, due to soft tissue compression, in order to limit movement of the knee to the desired radius. Ensure that the brace joints are in the correct position: The middle of the brace joints should be level with the top of the kneecap. The brace joint must be behind the lateral central line of the leg.

Care instructions

The brace cannot be worn in saltwater. If the brace is worn in chlorinated water, it should be rinsed out thoroughly with clean water afterwards. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the paddings and covers by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.

- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Avional, velours, polyester and PU-foam.

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

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