




&




The new and improved application aid for donning on and doffing **closed toe medical compression stockings** and tights in one product.

Magnide® 2in1 offers a smooth transition of the garment onto the patient's skin. It makes the application of medical compression garments safe and easy, minimizing its wear and tear. Magnide® 2in1 has flexible sizing to fit different sizes of limbs. This is particularly important for patients diagnosed with lower limb edema (swelling). Magnide® 2in1 is a simple solution for healthcare professionals to recommend and for care recipients to use in combination with medical compression therapy.

## Self-care







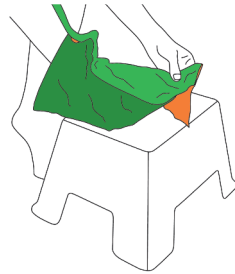


Don't push the foot to the end of the Magnide, leave some free space at the end





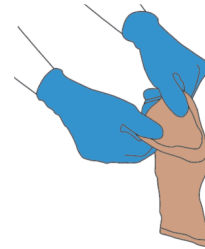


In case of a silicon border, make sure it's folded back before application

## Donning instructions

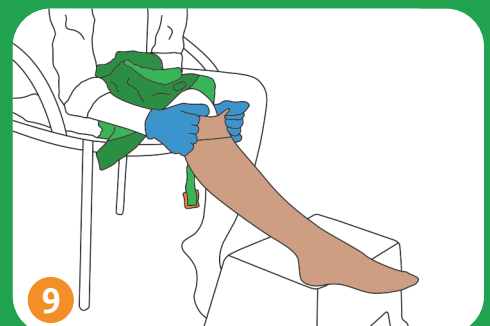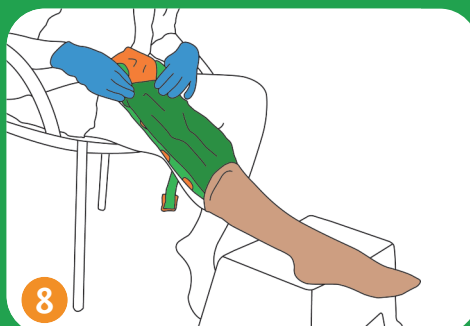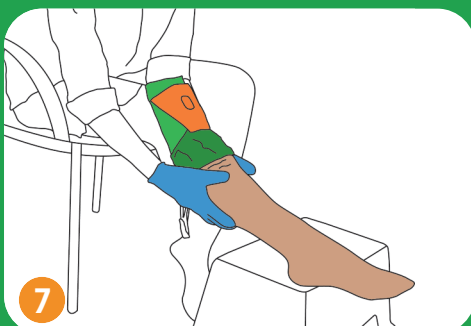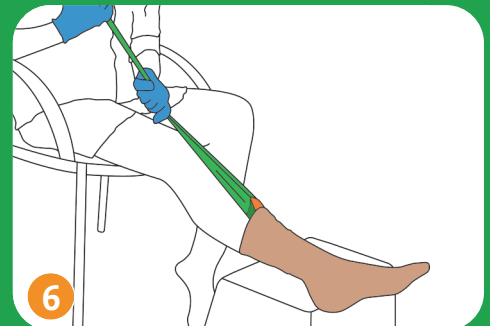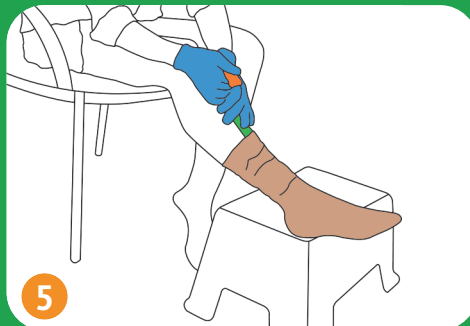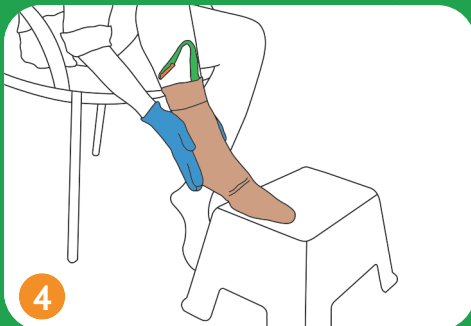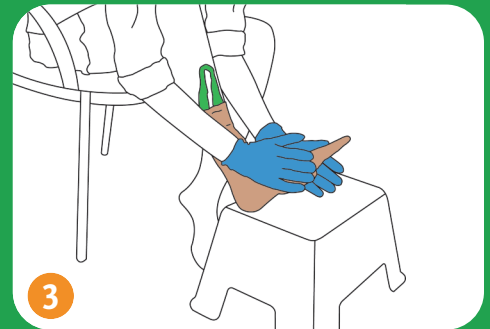


# Doffing instructions

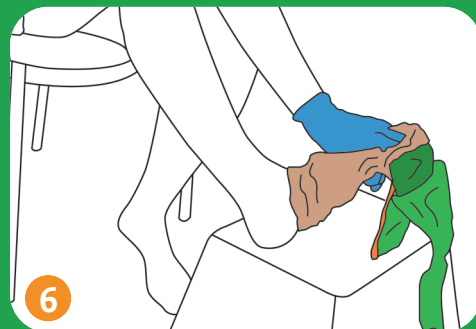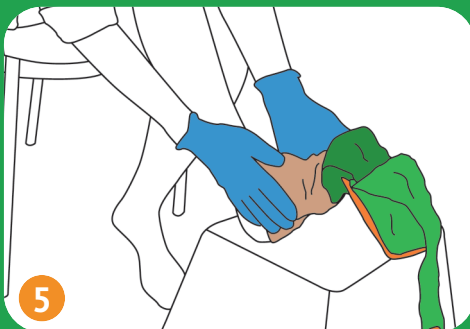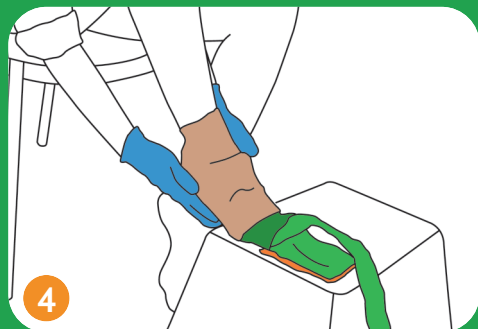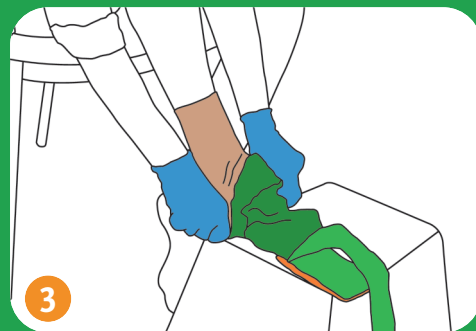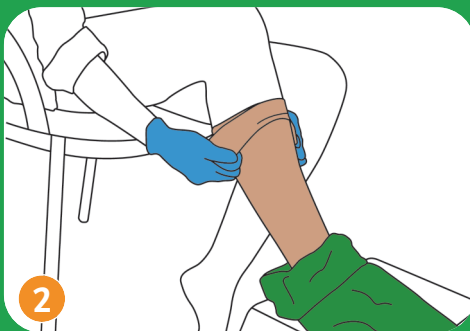

## Preparation for re-use


Fix the top of the loop to the top of the Magnide® by using the magnet

1

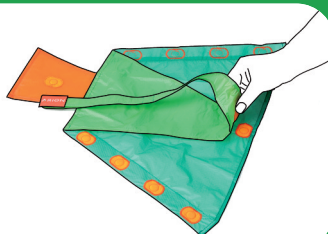


Turn over the Magnide® and place it on a flat surface

2

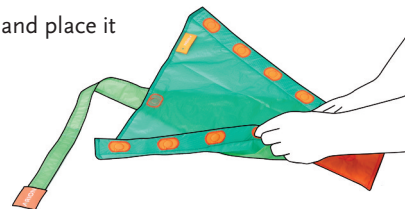


Fold both sides with magnets to the middle, so that the magnets make contact. The orange stitching is now visible on the outside

3

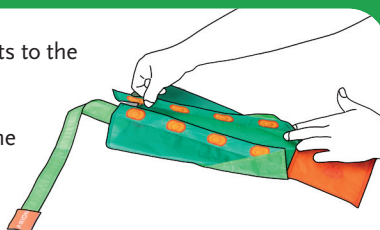

Close the orange flap (near the toes) over the Magnide® so that the magnets make contact

4

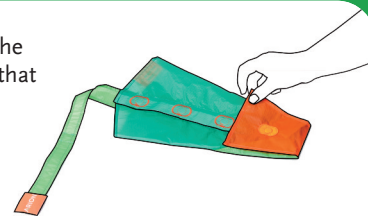

## ... We make your life easier!

### Easy to Use

- Minimizes friction between compression garment and skin
- Reduces daily stress during donning and doffing
- Increases independence of compression wearers

### Safe to Apply

- Reduces risk of skin damage; can be especially beneficial in case of high skin sensitivity, leg ulcers
- Extends life-span of compression garments
- Helps to avoid garment overstretching or distortion

### Simple Solution

- Offers application aids for all types of compression garments: stockings, sleeves, gloves
- Designed for different shapes and sizes of limbs
- Empowers concordance to compression therapy

### Sizes

Art. Nr.	Size	Shoe Size	
		Woman	Man
E01540	Medium	≤ 7.5	≤ 6
E01541	Large	8.5 - 11	6.5 - 10
E01542	X-Large	--	≥ 11

XL also suitable for extra wide feet with cA upto 11 inch

