

magnide®
easy, safe and simple



Donning




QR code magnide


The go-to donning aid for **closed toe medical compression stockings** and tights

Magnide® provides an effective solution for all wearers of medical compression stockings and tights with closed toes or their caregivers. Hosiery can now be donned and doffed with ease. Magnide® has flexible sizing to fit different limb sizes. This is particular important for patients diagnosed with edema.

Self-care






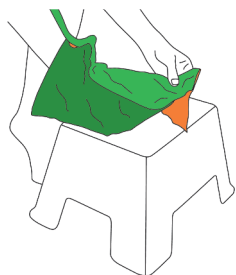


Don't push the foot to the end of the Magnide, leave some free space at the end

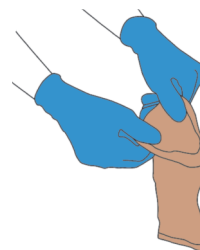

In case of a silicon border, make sure it's folded back before application

Donning instructions

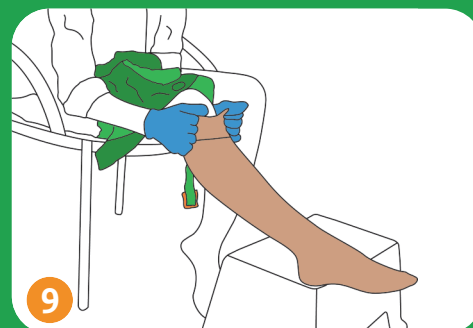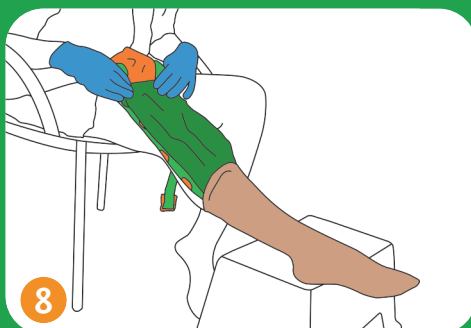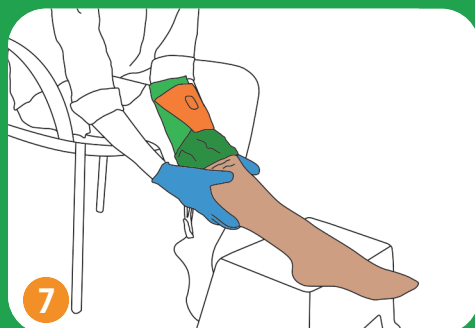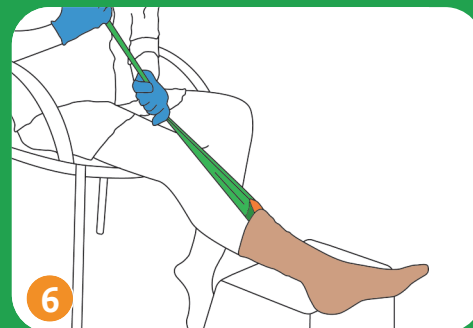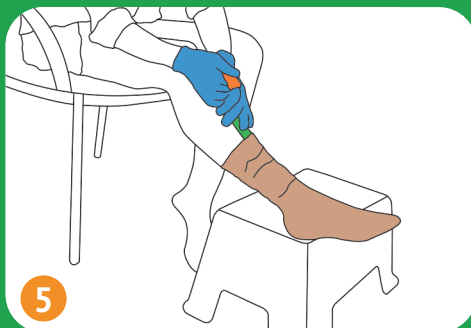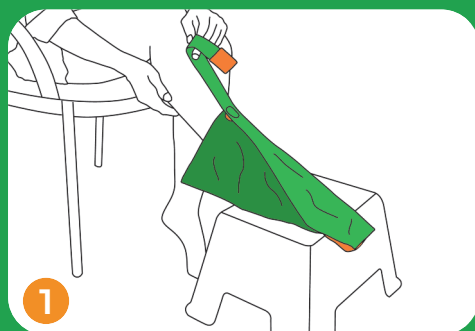


Fix the top of the loop to the top of the Magnide® by using the magnet

1




Turn over the Magnide® and place it on a flat surface

2

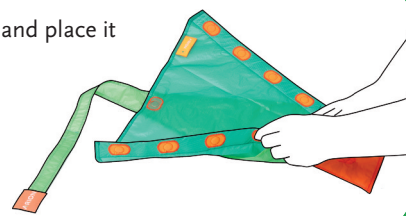


Fold both sides with magnets to the middle, so that the magnets make contact. The orange stitching is now visible on the outside

3

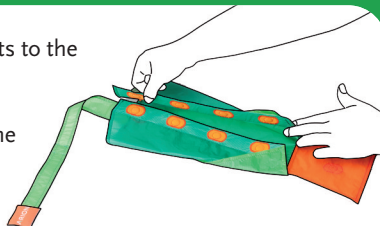

Close the orange flap (near the toes) over the Magnide® so that the magnets make contact

4

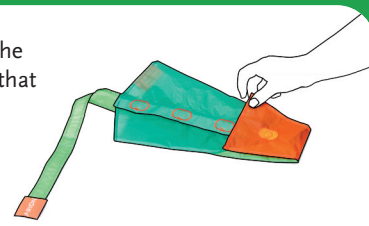

... We make your life easier!

Easy to Use

- Minimizes friction between compression garment and skin
- Reduces daily stress during donning and doffing
- Increases independence of compression wearers

Safe to Apply

- Reduces risk of skin damage; can be especially beneficial in case of high skin sensitivity, leg ulcers
- Extends life-span of compression garments
- Helps to avoid garment overstretching or distortion

Simple Solution

- Offers application aids for all types of compression garments: stockings, sleeves, gloves
- Designed for different shapes and sizes of limbs
- Empowers concordance to compression therapy

Sizes

| Art. Nr. | Size | Shoe Size | |
|----------|---------|-----------|----------|
| | | Woman | Man |
| E01513 | Medium | ≤ 7.5 | ≤ 6 |
| E01514 | Large | 8.5 - 11 | 6.5 - 10 |
| E01515 | X-Large | -- | ≥ 11 |

XL also suitable for extra wide feet with cA upto 11 inch

