

MIAMI LSO[™] & MIAMI TLSO[™]


Advanced compression and support

Össur's innovation in spine technology evolves beyond lumbosacral immobilization to include the thoracic segment, with the new pre-assembled Miami TLSO. The easy-to-use Miami line requires minimal preparation and is designed to provide optimal patient comfort. Easily applied in all configuration options, donning and doffing is further simplified with blue patient touch-points.



Easy for everyone

Pre-assembled design and intuitive features facilitate a more accurate sizing experience directly on the patient. Proven to be significantly superior in comfort and easier to don and doff*, Miami LSO & Miami TLSO's modular panel system is interchangeable to accommodate patient's support needs.

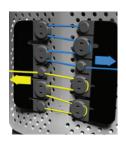

The proven power of the pulley

Advanced compression you can feel, with unmatched cinching travel distance backed by a double pulley system. Length adjustable pulley cords enable tailored placement and individualized compression.



Thin is in!

At just half the width of other belts, Miami LSO is discreet under clothing. And while thicker belts can retain heat, the ventilated belt promotes a breathable microclimate, helping to keep the skin cool and comfortable.



Blue is You™

Intuitive blue Patient Touch-Points and Quick-Release buckles send patients home with confidence.



^{*} Compared to Miami Lumbar TLSO



Miami TLSO™ 456



Miami TLSO™ 464 (Axillary straps not pictured)



Miami LSO™

PRODUCT HIGHLIGHTS

- 1 Anatomically contoured posterior panels
- 2 Blue is You™ patient touch-points
- 3 Thin, discreet belt
- Precise-Size™ landmarks
- 5 Fold-over belt arms with no trimming needed
- 6 Durable, ventilated fabric
- 7 Quick-fit pocket
- 8 Double pulley system with length-adjustable cords
- 9 Quick-release buckles
- Optional anterior-thoracic extension* with height accommodation from 5'1" 6'5" (155 cm 193 cm)
- Optional dual strapping (under & over arm configurations)
- 12 Telescoping Chest Strap with Quick-release buckle
- 13 Soft & Comfortable Shoulder Straps

INDICATIONS

Miami LSO: Conditions requiring gross immobilization of the trunk in the lumbar region. Such conditions may include: stable non-displaced spinal fractures (L1-L5), spinal stenosis, herniated disc, degenerative spinal pathologies, spondylolisthesis, and spondylolysis.

Miami TLSO: Conditions requiring gross immobilization of the thoracic and lumbar spine. Such conditions may include: post-surgical immobilization, spinal support up to T7, spinal stenosis, herniated disc, degenerative spinal fractures, spondylolisthesis, and spondylolysis.

^{*} Miami TLSO 464 only

ORDERING INFORMATION

OPTION 1: Order braces for inventory



STANDARD POSTERIOR PANEL		
SIZE	PART #	
Standard	MLSO-P56	

O-2	UNIVERSAL POSTE	ERIOR
O-1EXT		
O-2EXT		
	SIZE	PART #
	with Lateral	MLSO-P55
	without Lateral	MLSO-P57

ANTERIOR PANEL		
SIZE	PART #	
6" (Slim)	MLSO-PA6	
8" (Standard)	MLSO-PA8	

CONTOURED POSTERIOR PANEL SIZE WAIST CIRC. PART # Small (S) 26-34" (66-86 cm) MLSO-P11 Medium (M) 34-42" (86-107 cm) MLSO-P21 Large (L) 42-50" (107-127 cm) MLSO-P31 Extra Large (XL) 50-70" (127-177 cm) MLSO-P41

CONTOURED POSTERIOR PANEL WITH LATERAL SUPPORTS SIZE WAIST CIRC. PART # Small (S) 26-34" (66-86 cm) MLSO-P12 Medium (M) 34-42" (86-107 cm) MLSO-P22 Large (L) 42-50" (107-127 cm) MLSO-P32 Extra Large (XL) 50-70" (127-177 cm) MLSO-P42

OPTION 2: Order a brace in a kit

MIAMI LSO - PRE ASSEMBLED		
CONFIGURATION		PART #
Universal (select markets only)	626	MLSO-X56-2
	627	MLSO-X56-1
	631	MLSO-X57-1
	637	MLSO-X55-1
Contoured	631	MLSO-XY1-1
	637	MLSO-XY2-1

MIAMI TLSO - PRE ASSEMBLED			
CONFIGURATION		PART #	
Universal	456	MTLSO-X352	
(select markets only)	464	MTSLO-X351	
Contoured	456	MTLSO-X1Y2	
	464	MTLSO-X1Y1	
Contoured Non-Lordotic (select markets only)	456	MTLSO-X2Y2	
	464	MTLSO-X2Y1	

All pre assembled kits accommodate waist circumferences 26"-50".

Key: X = 1 (Slim belt), 2 (Standard belt)

Y = 1 (SM), 2 (MD), 3 (LG), 4 (XL), 5 (universal)

Z = 1 (Contoured), 2 (Contoured w/ lats), 5 (Universal w/ lats), 6 (Standard) or 7 (Universal no lats)

A = 1 (Anterior panel) or 2 (No Anterior)

ACCESSORIES	
6" slim belt extension*	MLSO-1EXT
8" standard belt extension*	MLSO-2EXT
TLSO Axillar pads and straps	MTLSO-0003
TLSO optional Hex tool	MTLSO-0004
Össur fit tool	B-040100000

^{*} For waist circumferences >50" - 70"

FOLLOW ÖSSUR ON











USA (800) 233-6263 WWW.OSSUR.COM CANADA (800) 663-5982 WWW.OSSUR.CA

