
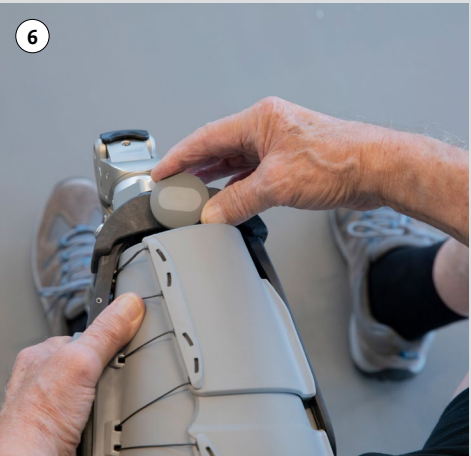
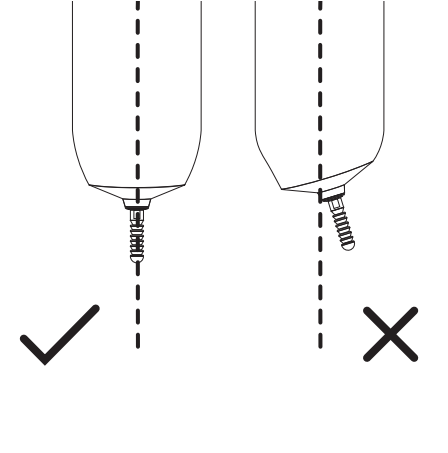


How to put on

- 1. Make sure you are in a seated position, preferably on a hard and smooth chair.
- 2. Grip the top of the liner from the inside and slide it over the hand until the inner surface is fully exposed, taking care not to damage it with fingernails.
- 3. After exposing as much of the bottom of the liner as possible, position it against the end of the residual limb. Make sure the pin is positioned in line with the residual limb. The longest side


should go to the outside of the residual limb, the shortest side to the inside of the residual limb. With light compression roll the liner all the way upwards onto the limb. **Do NOT tug or pull.**

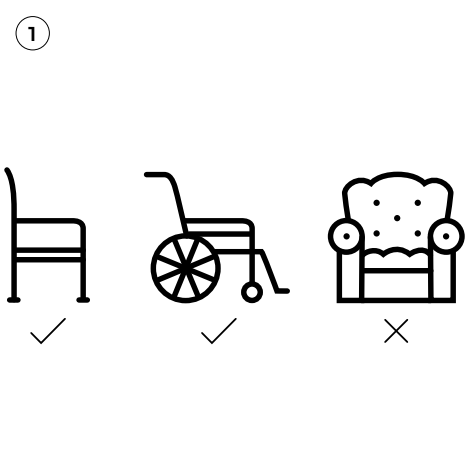

- 4. Open the socket by pulling the handle up, away from the knee.
- 5. Slide the socket over your residual limb, pull towards you, and push residual limb into the socket.
- 6. Push the pin into the hole in the bottom. The knob on the outside will turn if the pin engages the lock. Try to turn the knob to pull the pin into the lock completely.
- 7. Close the socket by pushing the handle completely down, towards the knee.

How to take off

- 1. Make sure you are in a seated position, preferably on a hard and smooth chair.
- 2. Open the socket by pulling the handle up, away from the knee.

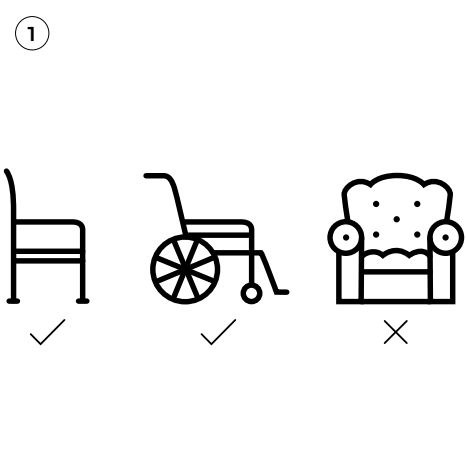

- 3. Pull up and hold up the knob. Slide the socket away from the residual limb and pull the residual limb out.

How to clean

Clean with a damp cloth and a mild soap. Or use disinfectant wipes to clean the device. Air dry, do not expose to direct heat.

**Do NOT remove the grey strut covers or adjust any screws.**



## CONNECT® TF

### Patient Guide

 Össur hf.  
Grjótháls 1-5  
110 Reykjavík  
Iceland

Össur Americas  
27051 Towne Centre Drive  
Foothill Ranch, CA 92610, USA  
Tel: +1 (949) 382 3883  
Tel: +1 800 233 6263  
ossurusa@ossur.com