2R8*, 2R31*, 2R54*

DE	Gebrauchsanweisung	3
EN	Instructions for use	8
FR	Instructions d'utilisation	14
IT	Istruzioni per l'uso	20
ES	Instrucciones de uso	26
PT	Manual de utilização	32
NL	Gebruiksaanwijzing	38
SV	Bruksanvisning	44
DA	Brugsanvisning	49
NO	Bruksanvisning	55
FI	Käyttöohje	60
PL	Instrukcja użytkowania	66
HU	Használati utasítás	72
CS	Návod k použití	78
RO	Instrucțiuni de utilizare	83
HR	Upute za uporabu	89
SL	Navodila za uporabo	95
SK	Návod na používanie	100
BG	Инструкция за употреба	106
TR	Kullanma talimatı	112
EL	Οδηγίες χρήσης	118
RU	Руководство по применению	124
JA	取扱説明書	130
ZH	使用说明书	136
KO	사용 설명서	141

1

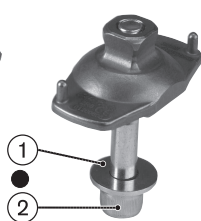
2R54

2R31

2R8

INFORMATION

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INFORMATION

Last update: 2016-01-22

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

1.1 Construction and Function

The foot adapters 2R8* (steel), 2R31* (titanium) and 2R54* (aluminium) are for connecting the prosthetic foot to the distal connection of the modular prosthesis.

1.2 Combination possibilities

This prosthetic component is part of the Ottobock modular system. As a result of its design, it may be combined with other prosthetic components of the modular system. You can find possible restrictions in this section. Contact the manufacturer with questions.

2 Application

2.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

2.2 Area of application

2R8=M8, 2R31=M8, 2R54=M8 and 2R54=M10

Approved for a body weight of **up to 100 kg**.

2R8=M10

Approved for a body weight of up to **125 kg**.

2R31=M10

Approved for a body weight of up to **136 kg (300 lbs)**.

2.3 Environmental conditions

Allowable environmental conditions
Temperature range for use: -10 °C to +60°C
Allowable relative humidity 0 % to 90 %, non-condensing
Unallowable environmental conditions
Mechanical vibrations or impacts
Perspiration, urine, fresh water, salt water, acids
Dust, sand, highly hygroscopic particles (e.g. talcum)

2.4 Service Life

In general, all modular adapters are tested by the manufacturer for 3 million load cycles. Depending on the patient's level of activity, this corresponds to a service life of 3 to 5 years.

3 Safety

3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

3.2 General safety instructions

 **CAUTION**

Excessive strain on the product

Risk of injury due to breakage of load-bearing components

- ▶ Use the prosthetic components according to the classification (see section "Area of application").

 **CAUTION**

Unallowable combination of prosthetic components

Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved according to the section "Combination possibilities".
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

 **CAUTION**

Use under unallowable environmental conditions

Risk of injury due to damage to the product

- ▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental conditions").
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).

 **CAUTION**

Exceeding the service life and reuse on another patient

Risk of injury due to loss of functionality as well as damage to the product

- ▶ Ensure that the approved service life is not exceeded (see section "Service life").
- ▶ Only use the product for a single patient.

⚠ CAUTION**Mechanical damage to the product**

Risk of injury due to change in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

4 Scope of delivery

2R8*, 2R31*, 2R54* Foot adapters		
Quantity	Designation	Reference number
1	Instructions for use	–
1	Foot adapter	–

Spare parts/accessories (not included in the scope of delivery)		
Quantity	Designation	Reference number
Ottobock special adhesive with hardener		636W28
		636W18 and 636W19
Single component pack for 2R8=M8		2D6=M8
Single component pack for 2R8=M10		2D6=M10
Each consisting of		
1	Washer	–
1	Cap screw (steel)	–
Single component pack for 2R31=M8, 2R54=M8		2D7=M8
Single component pack for 2R31=M10, 2R54=M10		2D7=M10
Each consisting of		
1	Washer	–
1	Cap screw (titanium)	–

5 Preparation for use

CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- Observe the alignment and assembly instructions.

5.1 Mounting the adapter

CAUTION

Improper assembly of the screw connections

Risk of injury due to breakage or loosening of the screw connections

- Clean the threads before every installation.
- Apply the specified torque values.
- Follow the instructions regarding the length of the screws and about how to secure the screws.

- > **Required tools and materials:** Grinding tool, 710D4 torque wrench, degreasing cleaning agent (e.g. 634A3 acetone), special adhesive and hardener (see Page 11, table of spare parts/accessories)
- 1) For prosthetic feet with wooden components, roughen the connection surface and remove the grinding dust.
 - 2) **INFORMATION: The foot adapter has to be glued to the prosthetic foot. Use the specified special adhesive for this purpose.**
Mix the special adhesive and hardener well together and apply to the connection surface.
 - 3) Clean any soiling (e.g. caused by grease) on the metal surface of the foot adapter to be glued with a degreasing cleaning agent.
 - 4) Position the foot adapter on the prosthetic foot, insert the screw and washer from below and tighten it (installation torque: **2 to 4 Nm**).
 - 5) After **3 to 4 hours**, tighten the screw connection with the required installation torque (see table).
 - 6) Leave the glued surface to harden (**16 hours at 20 °C** – the time increases for lower temperatures).

Reference number	Prosthetic foot size [cm]	Torque [Nm]	
		Prosthetic feet with wooden components	Prosthetic feet with plastic components
2R8=M8, 2R31=M8, 2R54=M8	All	20	–

Reference number	Prosthetic foot size [cm]	Torque [Nm]	
		Prosthetic feet with wooden components	Prosthetic feet with plastic components
2R8=M10, 2R31=M10, 2R54=M10	21-25	20	50
	26-30	30	50

6 Maintenance

- ▶ The prosthetic components should be inspected after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

7 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

8.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the

material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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Otto Bock HealthCare GmbH