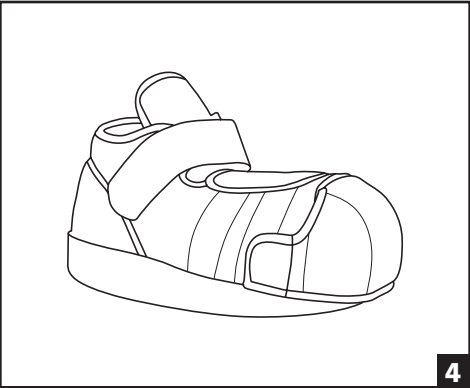
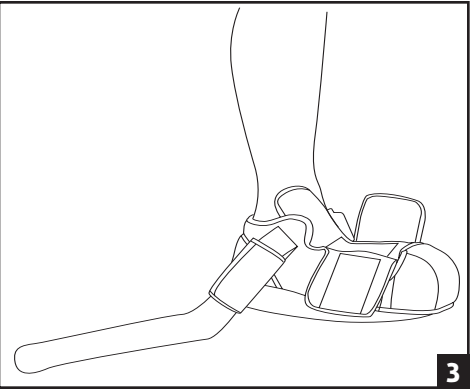
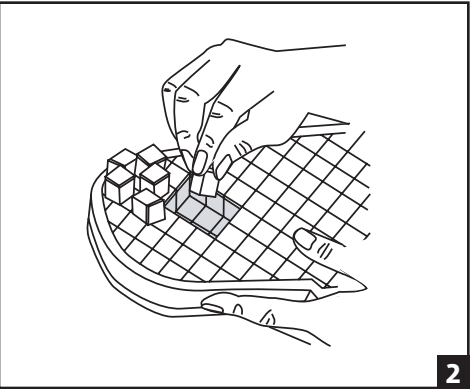
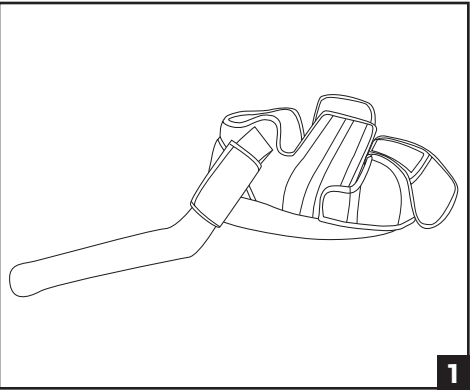
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: The Procure Off-Loading Diabetic Shoe is designed to provide support and protection to the foot and help in the prevention of diabetic foot ulcers. Providing mild protection of a limb or body segment for acute and prophylactic care.

CONTRAINDICATIONS: If bony foot deformities exist, such as non-operative Charcot or other abnormalities, that cause the shape of the foot to deviate excessively from its natural shape. Foot pathology requiring support of the boney structures.

WARNINGS AND PRECAUTIONS:

- Not intended for patients in excess of 300 lbs.
 - Never allow the bare foot to directly contact the innersole.
 - Wear a sock to allow the foot to breathe.
 - Patient, family member or medical personnel should inspect your feet before and after each use.
 - If you develop any red inflamed areas, ulcers, sores or draining areas on your foot, contact your physician immediately.
 - Wounds located at the beginning or the end of the gait path should be monitored closely to ensure that pressure relief is maintained.
 - The insole should NOT be used in conjunction with any other product other than the Off-Loading Diabetic Shoe.
 - Do not use over open wounds. This may result in reduction of blood flow or sensation.
 - Do not over tighten straps. This may result in reduction of blood flow or sensation.
 - Do not modify the device or use the device other than intended.
 - Do not use this device if it is damaged and/or the packaging has been opened.
 - Do not use this device on patient's incapable of communicating physical discomfort.
 - If pain, swelling, changes in sensation or other unusual reactions occur while using this product, consult your medical professional immediately.
 - If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

- PREPARATION:**
- Be sure to treat and dress the plantar ulcer(s) appropriately.
 - Wear a sock over the foot and dressing(s)

OFF-LOADING DIABETIC SHOE APPLICATION

1. Unfasten and loosen closures. **(Fig. 1)**
2. Remove the blue foam pad from the shoe. Place the blue side against the patient's plantar surface, locate and remove the pre-cut foam cubes from the bottom of the blue foam pad around the wound site(s). Replace the foam pad back into its original location in the shoe. **(Fig. 2)**
3. Slip foot into the shoe. **(Fig. 3)**
4. Close shoe: apply tongue first along length of the foot, close both side of the shoe on top of the tongue, lock shoe with adjustable strap around the ankle. **(Fig. 4)**
5. The toe cover can be removed if indicated by your physician.

CLEANING INSTRUCTIONS: To clean shoe and blue foam, hand wash in cold water using mild soap, or wipe with a damp cloth and air dry. Note: if not rinsed thoroughly, residual soap may cause irritation and deterioration of materials.

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