

## BRACES & SUPPORTS



TEMPERATURE



NO BLEACH



NO TUMBLE DRY



NO IRONING



NO DRY CLEAN

## SLEEVES



TEMPERATURE



NO BLEACH



NO TUMBLE DRY



NO IRONING



NO DRY CLEAN

## SOCKS



TEMPERATURE



NO BLEACH



LOW HEAT TUMBLE



NO IRONING



NO DRY CLEAN

## PLEASE FOLLOW OUR WASHING INSTRUCTIONS FOR BEST RESULTS AND LONGEVITY

### Washing

- You may machine wash your support, but hand washing may help extend the life of your product
- Ensure any velcro tabs are secured to their straps, not hanging loose or attached to the knit, which can snag and cause runs in the fabric
- Machine wash in cold/cool water (~ 86°F), using the gentle or delicate cycle setting
- Wash with similar colors using any regular laundry detergent with no additives (no fabric softener or Woolite®)
- We recommend using a mesh washing bag

### Drying

- Lay your support flat to air dry, do not hang it to dry as it may stretch out
- Do **not** dry it your Bauerfeind support in a dryer, this can compromise the knitted fabric

Please wash your Bauerfeind supports regularly to retain the compression of the knitted fabric!