

# +REPAREL™

from CHALLENGERHEALTH®



CELLULAR  
STIMULATION



UNMATCHED  
COMFORT



INCREASES  
CIRCULATION



ALLOWS FULL  
RANGE OF MOTION

THE EVOLUTION  
OF INFLAMMATION  
MANAGEMENT

## Our Products

The Reparel™ line of non-compressive sleeves is designed to promote the resolution of the inflammatory response without utilizing the mechanical 'squeeze' typical of compression garments. Our innovative fabric is both soft and breathable—a far more comfortable option for reducing pain and resolving inflammation.

**Joint Sleeves** – superior to common neoprene sleeves, these are designed to address acute and chronic inflammation of supporting joint tissue by improving circulation without interfering with your daily activities.

**Specialty Medical Sleeves** – a substitute for commonly prescribed, graduated compression hosiery used post-surgery. They are designed to act as a comfortable and functional post-operative sleeve, either alone or as an undersleeve in conjunction with a brace to both protect and promote healing.

## Our Technology

Reparel™ fabric incorporates semi-conductive nanoparticles as part of the manufacturing process. These nanoparticles convert body heat (thermal energy) into photonic energy, a proven vasodilator that has been shown to enhance cellular metabolism, resulting in improved tissue repair, faster resolution of the inflammatory response, and reduced pain.

## Indications


- + **Swelling** (acute and post-operative)
- + **Repetitive stress injuries**
- + **Sports injuries & tendinitis**
- + **Degenerative joints**



## Application *Use Reparel for:*

- + **Non-Operative Relief**  
of Pain/Inflammation in Major Joints
- + **Effectively Resolving**  
the Inflammatory Response Post-Surgery
- + **A Dual-Purpose Undersleeve**  
with Every Functional Brace Prescription (billable)

## Recommended Protocol: Post-Surgical Care for Knee Procedures

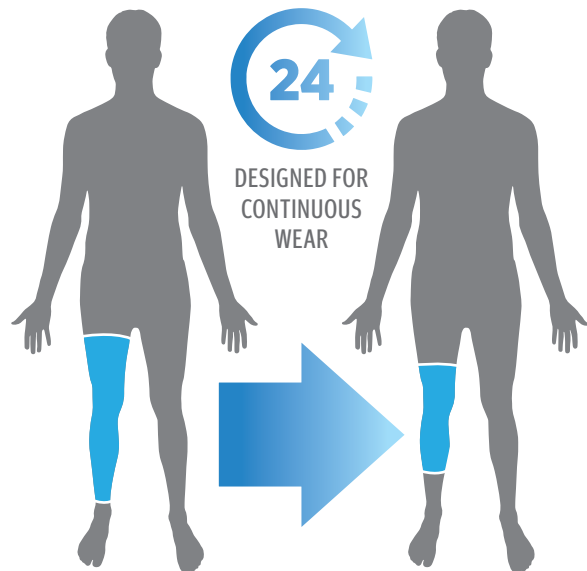

### LEG SLEEVE 7-10 DAYS POST-OP

When patient becomes more ambulatory, the surgery site is wound stable, and the greatest risk period for DVT has passed. Patients should wear until swelling decreases (approximately 4-6 weeks; up to 8 if swelling is persistent), becomes more localized and allows for full range of motion.

### KNEE SLEEVE 5-8 WEEKS POST-OP

Apply as soon as patient is out of Full Leg Sleeve (approximately 5-8 weeks post). Patients should wear as needed to combat intermittent pain and stiffness as they return to full activity level

**+REPAREL™**  
from CHALLENGERHEALTH®

## Pricing and Billing Reparel™

In response to increasing administrative costs, ever-changing guidelines for insurance reimbursement, and consistent patient requests for trusted and affordable solutions, we offer our physicians a unique hybrid model designed to directly benefit your bottom line while reducing your patients' out of pocket costs.

Reparel Knee and Full-Leg Sleeves can be billed using separately payable addition code L2397 (addition to lower extremity orthosis – suspension sleeve) when prescribing functional knee braces.

Billing the Knee and Leg sleeves allows your practice to both eliminate a cost to your patient and generate enough revenue to provide added flexibility when setting the retail price of Reparel's cash-only products: the Ankle, Elbow, and Wrist sleeves, as well as the Knee Sleeve (when it is prescribed without an orthosis).

Reparel sleeves are available exclusively through our physician customers and [www.reparel.com](http://www.reparel.com). Supporting our customers and their interactions with their patients is our most valued goal, and for that reason we maintain a very strict, fixed pricing policy for our online sales. You'll never receive competition from us.

## Implementing Reparel in Your Practice

Bringing Reparel into your practice is both simple AND risk-free. Our minimum opening order is only \$250, and every new clinic carrying Reparel has a full 60 days to gauge receptiveness and narrow the specific product mix that best suits your patient population. Any time during those 60 days you may return your Reparel inventory for a full refund with no associated fees. Additionally, as long as you're carrying Reparel within your practice we will guarantee every sleeve for every patient. Should a patient return claiming the sleeve did not deliver the outcome they were promised, we will credit your account the cost of that item\*

\*certain restrictions apply

## Reparel Commitment

Reparel shares a simple but very important goal with every one of its physician customers: to provide the best possible patient care. Reducing pain, and discomfort caused by inflammation is a critical step in producing better outcomes, and until now there have been very few innovative products available to effectively address these issues. Patients deserve an effective, comfortable, less intrusive solution, and Reparel makes it possible for you to provide it to them.

**FOR ORDERING INFORMATION,  
PLEASE EMAIL US AT  
[SALES@REPAREL.COM](mailto:SALES@REPAREL.COM)  
OR CALL 530.809.1172**



**[REPAREL.COM](http://REPAREL.COM)**