## Rolyan® TakeOff® Therapeutic Forearm Band:

A808-460

A

B

### INDICATIONS

- To help reduce pain caused by repetitive motions

### CONTRAINDICATIONS

- Not for use directly over open wounds and infected areas
- May not be for people with circulatory problems
- Not for children who may be experiencing growth plate problems or people with elbow instability

### PRECAUTIONS

- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- The strap should fit snugly and comfortably without interfering with circulation.
- Wearing the strap while sleeping is not recommended because elbow positioning may cause restriction of blood flow.

### CARE AND CLEANING

Prior to cleaning, remove the pad from the strap and fasten hook closures.

#### Gel Shell® Pad

- Hand wash in warm water (hot water may yellow the pad) and mild soap. Rinse thoroughly. Press gently to remove excess water and let it air dry. Be sure the pad is completely dry before reapplying it.

#### Strap

- For extended use, soak the fabric for five minutes in a solution of mild detergent and cold water. Rinse the detergent out completely. Do not wring. Allow to air dry thoroughly before reuse.

### INSTRUCTIONS FOR USE

1. To attach the optional pad to the strap, position the strap with the foam side facing up. Attach the hook material on the pad to the loop material on the strap.

2. Remove the protective plastic sheet from the surface of the gel pad.
3. To apply the strap, with the foam toward the inside, put the hook end of the narrow portion of the strap into the slit, making a loop.
4. Extend the arm with the thumb pointing up and a slight bend in the elbow joint.
5. With the foam against the skin, slide the strap onto the forearm and position the top of the strap 1" to 1½" (2.5–3.8 cm) below the elbow joint. If the pad is used, position it so it applies localized pressure to the affected muscles (either the extensor or flexor muscles).
6. Pull the narrow end of the strap through the slit (A) and, foam side toward the strap, wrap it around the arm, fastening it to the strap.
7. Wrap the wider portion of the strap, foam side in, around the arm and fasten the hook closure.
8. A properly applied TakeOff Therapeutic Forearm Band is shown in B.

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**Patterson Medical Ltd.**