

Instructions for Use



- Ensure all closures, straps and accessories are secured correctly according to the included directions.
- Follow all written instructions included with the product, along with all verbal instructions and advice provided by your physician or care provider.

Each Rooke AKA Rigid Dressing Includes:

- 1 AK Protector
- 2 Distal spacer
- 3 One waist belt
- 4 Two limb attachment straps (2nd strap to allow for bifurcated attachment)
- 5 Scissors for trimming

Limb Placement and Fit:

1. Remove all the items and the spacer. Open the AKA Rigid Dressing and place it around the residual limb.



2. Approximate for trimming and trim excess material as needed. Cut between the double-stitched lines.

Note: On a short residual limb, insert spacer.



3. Close for a gentle, snug fit around the residual limb.



Rooke AKA Rigid Dressing

Instructions continued on reverse

Washing instructions: Wash in cold water 30C/85F, gentle/delicate cycle, line dry. DO NOT: Bleach, dry clean or iron.

Waist Belt Attachment:

1. Place the waist belt around the patient's waist (above any Pannus) and adjust to a gentle, snug fit.



2. Take one limb attachment strap and slide it UNDER the waist belt with the white hook Velcro facing OUT.



3. Fold the limb attachment strap OVER the waist belt and secure it to itself.



4. Secure the limb attachment strap to the AK Protector.

