Ulna Fracture Brace and S-Series

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Material Composition: 45% Low Density Polyethylene Plastic, 45% Polyethylene Closed-cell foam, 7% Nylon hook and loop strap, 3% nylon fastener.

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ENG

Purpose of Product: The Ulna Fracture Brace provides treatment of mid shaft and distal ulnar fractures, while allowing full range of wrist and elbow motion.

Indications:

- Treatment of mid shaft and distal ulnar fractures
- Post external fixation for healing of soft tissue
- Adjunct to internal fixation, such as unreamed intermedullary nails

Contraindications:

- Broken skin or superficial wounds
- Allergy to component

Side Effects: No known side effects with correct use

Fitting Instructions:

1. Apply talcum powder sparingly to your lower arm. Apply one two-ply polyester stockinette. Remove all wrinkles.
2. Loosen straps and apply brace by sliding hand and forearm through opening of brace. Note that foam lined shell should be positioned on outside (dorsal) of arm. Position brace just above bony prominence on the outside of wrist (styloid process).
3. Hold your arm firmly against your body and tighten lower strap, note that foam lined shell is overlapped by unlined shell. Then tighten upper strap. Equal pressure should be maintained on all straps. The velcro straps may be adjusted to comfort and maintain a snug fit.
4. The brace should fit snugly, but not be uncomfortable.
5. Bottom shell should overlap top shell. Overlap should be equal on both sides and will maintain alignment.
6. DO NOT REMOVE the Ulna Fracture Brace unless authorized by your physician.
7. Use your hand, wrist, elbow, and shoulder as much as possible.
8. If you have any questions or problems with your brace, contact your physician immediately.

Removal Instructions:

- For removal of UFB, hold your arm firmly against your body, loosen all velcro straps. Slide the outside and inside shell of the brace from your lower arm. Remove the stockinette.

Precautions: Follow the advice of the healthcare practitioner who prescribed or supplied the product. Wearing this product, as with any immobilizer product, could cause irritation (redness). Should this occur, readjust, or remove the product and contact your healthcare practitioner. Do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care Instructions:

- Clean brace with mild soap and water. Wash and dry stockinette, as necessary. Keep this instruction leaflet

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