




# SP-MAG

## Magnetic Wand

Replacement Magnetic Wand for the SmartPuck™ and ZeroPuck™

Parts Included

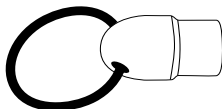

Replacement Magnetic Wand

1-year warranty against manufacturer defects, excessive wear or breakage.

External Prosthetic Components



SP-MAG.revA.06102025



Manufactured by Coyote®  
419 N. Curtis Rd., Boise, Idaho 83706 USA  
(208) 429-0026 | [www.coyote.us](http://www.coyote.us)





## ATTENTION

The SP-MAG is the Replacement Wand for the SmartPuck and ZeroPuck.

### Magnetic Wand

If the user does not intend on using the SmartPuck™ Smart Device Application or an iDevice is not available, the SmartPuck's™ internal pump can still be activated using the Magnetic Wand. The Wand allows the user to turn the SmartPuck's™ pump on and off by simply waving the wand along the side of the Socket. Bringing the magnetic wand to the side of the socket while the pump is enabled will disable the pump. Performing the same task while the pump is disabled will activate the pump.

### HALL SWITCH OPERATION (Magnetic ON/ OFF Switch)

#### LOCATION

1) The Hall switch position is marked with a sticker on the side of the SmartPuck™. The Dot should be positioned anterior /medial corner of the socket (for easy access and line of sight). The actual switch is located in the middle of the puck (measure up 3/4" from the bottom of the Puck).

#### OPERATION

1) Slide the magnet over the side of the Puck in the location described above in one single fluid motion. This motion will turn the pump ON.

2) Repeat the same procedure to turn the Puck OFF

**The Puck will vibrate 1X = SmartPuck™ is ON.**

**The Puck will Vibrate 2X = SmartPuck™ is OFF.**

#### VACUUM LEVEL

The SmartPuck™ will default to the last vacuum setting that was entered into the Puck. The SmartPuck™ will remember and maintain the last vacuum setting that was entered into the system.

See your Mobile APP under the ABOUT TAB for a detailed video on how to use this function.

### RESET SMARTPUCK™ (if having trouble with Bluetooth Connection)

1) Turn off SmartPuck™ with Magnetic Wand.

2) Hold Magnetic Wand over SmartPuck™ Hall Switch hearing one buzz.

3) Keep Magnetic Wand over the Hall Switch for 10 seconds until you hear a double buzz and the SmartPuck™ turns on. You have reset the SmartPuck™.



**See instructions online.**

[www.coyote.us/instructions](http://www.coyote.us/instructions)

EN | Instructions for Use

DE | Gebrauchsanweisung

FR | Notice d'utilisation

ES | Instrucciones para el uso

IT | Istruzioni per l'uso

NO | Bruksanvisning

DA | Brugsanvisning

SV | Bruksanvisning

NL | Gebruiksaanwijzing

RU | Инструкция по использованию

JA | 取扱説明書

ZH | 中文说明书

KO | 사용 설명서

FI | Käyttöohjeet

CS | Návod k použití

EL | Οδηγίες Χρήσης

PT | Instruções de Utilização

PL | Instrukcja użytkowania

TR | Kullanım Talimatları

