



sim-slide®
easy, safe and simple



&




QR code sim-slide


The optimal multi-functional application aid for donning and doffing **open toe** medical compression stockings and tights.

Sim-Slide® makes the application of medical stockings easier, thus reducing daily stress associated with donning and doffing compression garments. A Study shows that, when using Arion application aids, medical compression wearers achieve greater satisfaction with successfully donning and doffing their compression garments. This contributes to their positive user experience, thus improving therapy compliance and optimizing therapy results.

Self-care


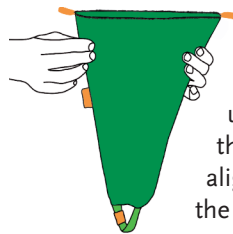


Make sure the product is ready for use by folding back the extra fabric and aligning the top with the product label







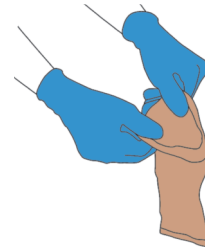


In case of a silicon border, make sure it's folded back before application

Donning instructions

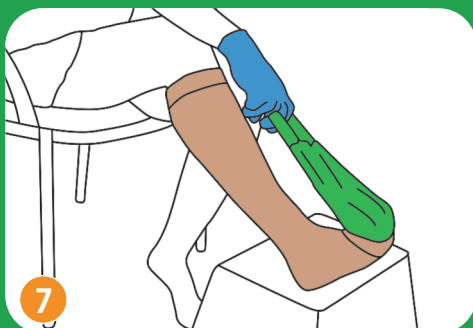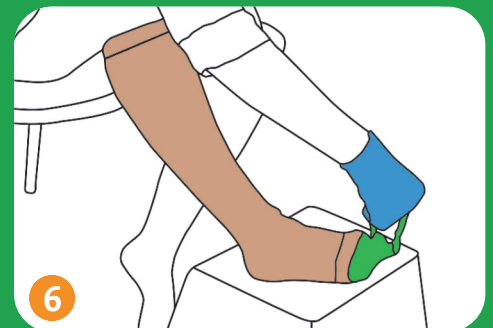

Doffing instructions

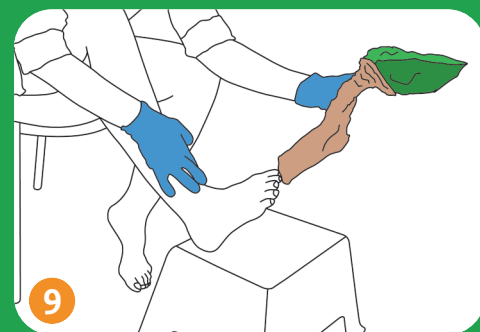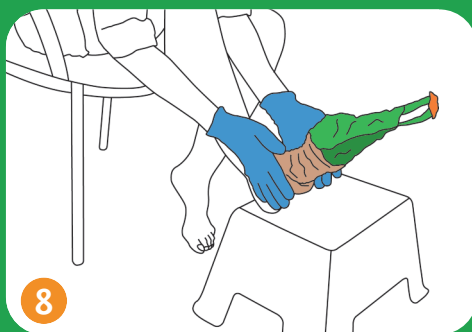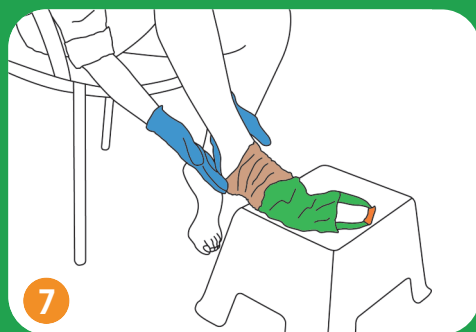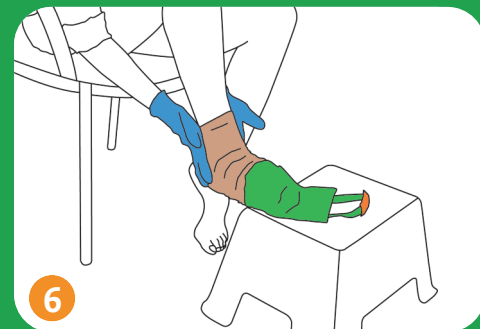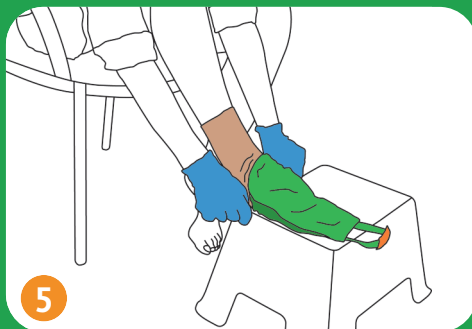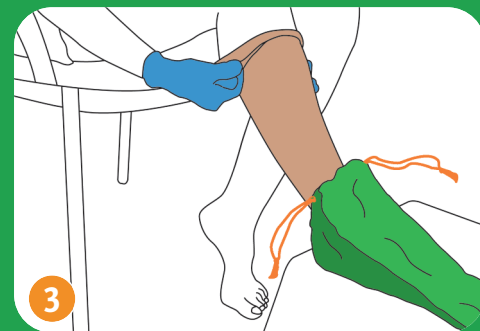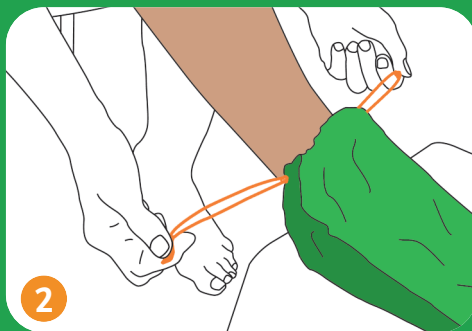

... We make your life easier!

Easy to Use

- Minimizes friction between compression garment and skin
- Reduces daily stress during donning and doffing
- Increases independence of compression wearers

Safe to Apply

- Reduces risk of skin damage; can be especially beneficial in case of high skin sensitivity, leg ulcers
- Extends life-span of compression garments
- Helps to avoid garment overstretching or distortion

Simple Solution

- Offers application aids for all types of compression garments: stockings, sleeves, gloves
- Designed for different shapes and sizes of limbs
- Empowers concordance to compression therapy

Sizes

Art. Nr.	Size	Shoe Size	
		Woman	Man
EoSlo1	Small	≤ 4 - 6	≤ 3 - 4.5
EoSlo2	Medium	6.5 - 8	5 - 6.5
EoSlo3	Large	.5 - 9.5	7 - 8
EoSlo4	X-Large	≥ 10	≥ 8.5