



QR code Slidex  
(Calm Stocking)




QR code Slidex  
(Thigh Stocking)

## Unique extension aid above the ankle

SlideX provides an effective solution for donning and doffing medical compression stocking, high tights and tights in case of high skin sensitivity, leg ulcers and/or irregular sizes/forms of limbs for patients and caregivers. Shall be used in combination with other Arion products such as the Easy-Slide, Sim-Slide®, Magnide® and Magnide® 2in1.

### Self-care (with Magnide®)






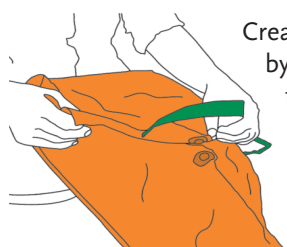


Create a double layer  
by folding the top of  
the slidex and  
connecting the  
magnets




Make sure you lay the SlideX  
on top of the leg with  
the folded end facing  
down and the handle  
facing upwards


## Donning instructions

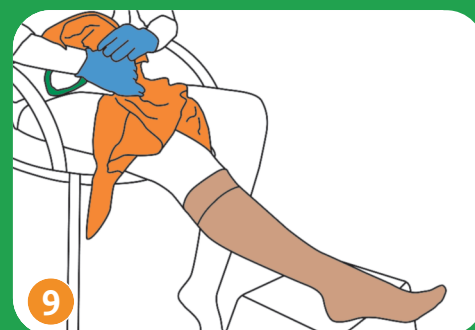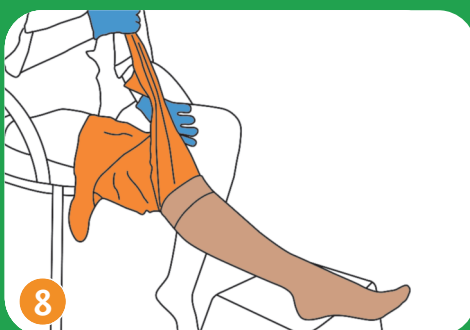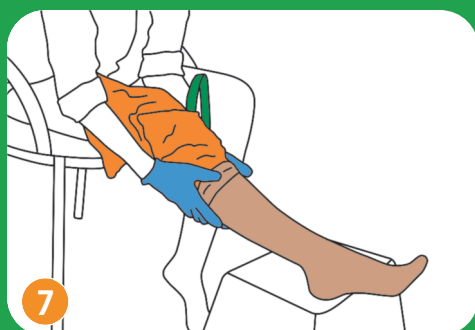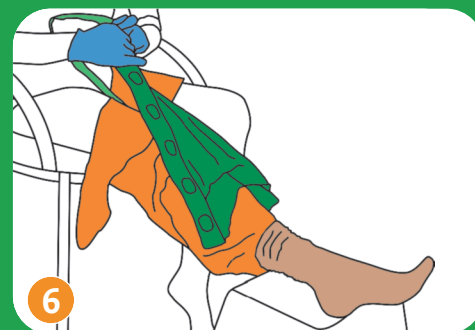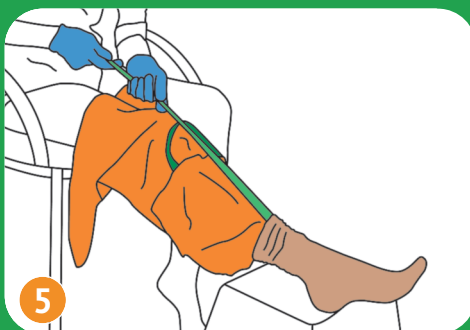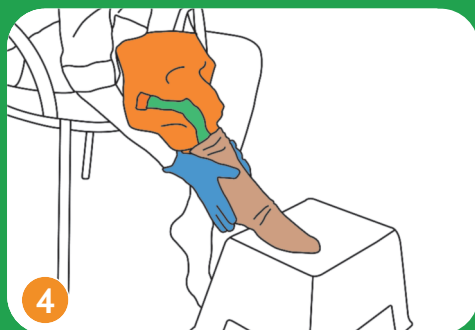








## Nursing (with Easy-Slide)

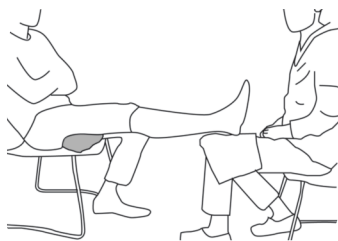

Make sure you're sitting in front of the patient, with the patients leg resting on your upper leg

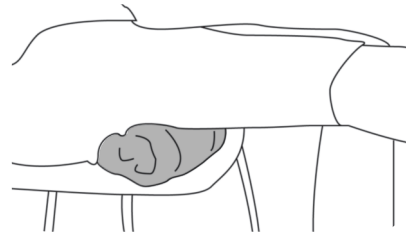

To avoid additional pressure, place a rolled towel underneath the patients upper leg

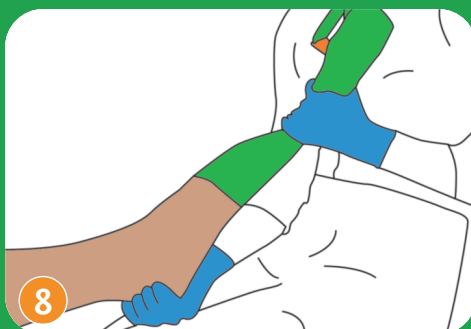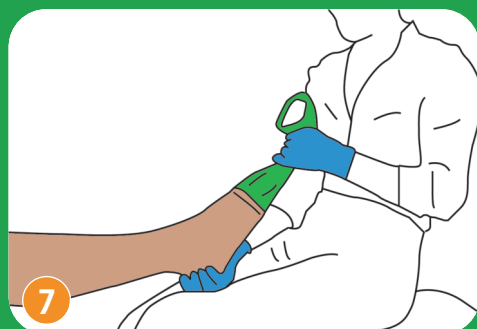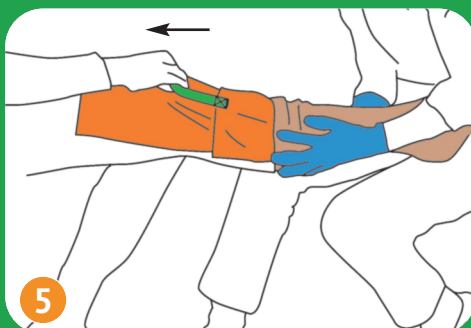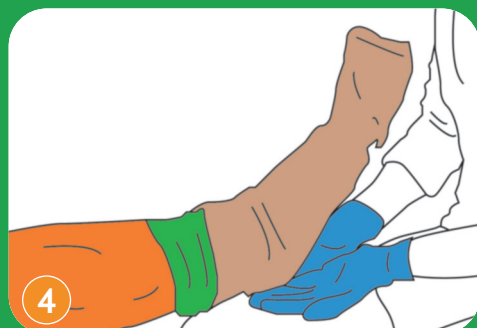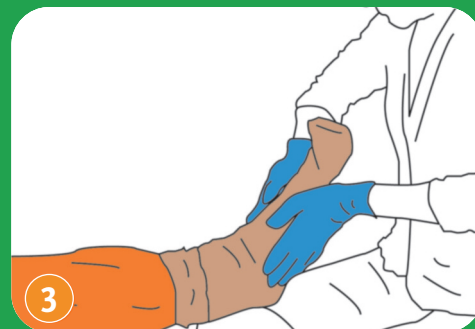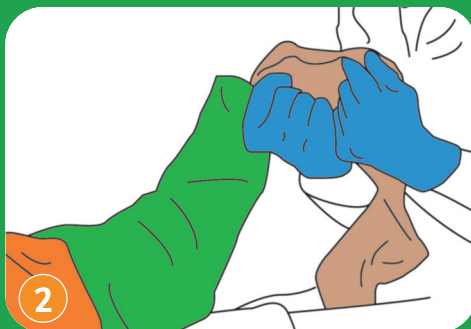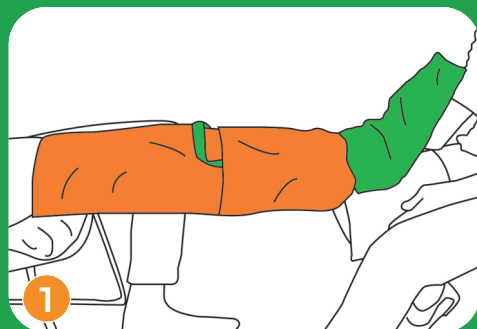

### Easy to Use

- Minimizes friction between compression garment and skin
- Reduces daily stress during donning and doffing
- Increases independence of compression wearers

### Safe to Apply

- Reduces risk of skin damage; can be especially beneficial in case of high skin sensitivity, leg ulcers
- Extends life-span of compression garment
- Helps to avoid garment overstretching or distortion

### Simple Solution

- Offers application aids for all types of compression garments: stockings, sleeves, gloves
- Designed for different shapes and sizes of limbs
- Empowers concordance to compression therapy

### Sizes

Art. Nr.	Size	Circumference of the lower leg
E01520	Medium	≤ 15,75 inch
E01521	Large	≤ 23,50 inch
E01522	Extra large	≤ 39,50 inch