AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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en english

These instructions for use can also be downloaded from:
www.bauerfeind.com/downloads

Dear Customer,

thank you for choosing a product from Bauerfeind. SofTec Genu is a high-quality medical product. Please read the **instructions** carefully and if you have any questions contact your doctor, medical retailer or our Technical Service department.

Intended use

SofTec Genu is a licensed therapeutic device designed to aid your treatment. It helps to protect your knee during therapy and assists your muscle development during physical therapy exercises.

 **SofTec Genu must be individually adapted to the patient by a trained orthopaedic technician in order to gain the maximum effect and optimal wear comfort from this orthosis¹.**

Indications

- Anterior cruciate ligament rupture (anterior instability)
- non-surgical treatment (conservative treatment)

- care of the knee before and after an operation
- long-term treatment for lasting anterior instability
- Conservative treatment for collateral ligament injuries
- Treatment for mild instability of the knee as a result of
 - advanced joint wear
 - (rheumatoid arthritis)
 - knee replacement (implant)
- Restricting the knee's range of motion e.g. after meniscal suturing or meniscal replacement (implantation)

Risks of using this product

Important Information

- SofTec Genu is a prescribed product that should be applied under a physician's guidance. SofTec Genu should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- Before applying SofTec Genu after an operation, be sure that the sensitivity and swelling are reduced enough to ensure that there are no unnecessary pressure points or complications. Typically, SofTec Genu is not applied post-operatively for 5–8 days, depending on the individual patient's pain and swelling.
- Modifications of SofTec or any joint bar adjustments may only be made by suitably trained specialists². Failure to comply with this requirement may adversely affect product performance, thereby cancelling any product liability.
- If SofTec Genu is used during exercise or sporting activities do not limit the flexion of the hinge. It is possible to limit extension to 0°, to avoid overstressing the knee joint.
- SofTec Genu is not designed to prevent injuries, especially when engaging contact sports. Contact sports inherently pose a greater risk of injury.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical advisor without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your medical advisor.
- When resting, the straps can be loosened for better comfort.
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Side effects, that affect the entire organism, have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

 Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Take off the orthose if you notice any allergic reaction.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date.

In the following conditions such aids should only be fitted and worn after consultation with your medical advisor:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
2. Varicose veins
3. Impaired sensation and circulatory disorders of the legs / feet, e.g. diabetes mellitus
4. Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid

Application instructions

Application and Removal of SofTec Genu:

- Open the zipper and all the straps. Pull the straps out of the plastic loops. Fasten down the ends of the straps on the Velcro area.
- Hold SofTec Genu with both hands at the upper end (with the label). Turn SofTec Genu so that the rear part (zipper) points to the front. (Fig. 1).
- Pull SofTec Genu over your ankle until the lower end is above your ankle.
- Turn SofTec Genu by 180° (half a turn) around your leg. (Fig. 2).
- Pull SofTec Genu up your leg until your kneecap is centered in the shaped ring (Fig. 3).
- The zipper can be closed easily by pulling the front lower part of SofTec Genu in the area of the zipper to the back until it overlaps. (Fig. 4).
- Guide all the straps through the plastic loops and fasten them loosely. An intended pressure is caused by tightening the straps in the following order. (Fig. 5):
 - Calf strap furthest from the knee (1)
 - Thigh strap closest to the knee (2)
 - Thigh strap furthest from the knee (3)
 - Calf strap closest to the knee (4).
- Be sure SofTec Genu is seated properly. Any straps that are too tight should be loosened. If you feel any pressure points in the area of the hinged side bars, they may need to be contoured by the orthopaedic technician who is treating you.
- Remove SofTec Genu by doing the above strapping sequence in the reverse order.

Cleaning instructions

⚠ Never expose the orthosis to direct heat (e.g. heater, sunlight, etc.)!

SofTec Genu can be washed separately by hand with a mild detergent at a temperature of 20–30 °C (68 °F–86 °F). The hinged side bars must be removed from the side pockets. This is done as follows:

- Open the two high straps. Pull the straps out of the plastic loops. Fasten down the ends of the straps on the Velcro area. The side pockets can be opened by pulling the entire Velcro section to one side. (Fig. 6).
- Remove the hinged side bars. **⚠ Important:** bear in mind that the joint bars are contoured to fit the body and are therefore different. The hinged side bars and the corresponding side pockets are marked with the same color so that the hinged side bars can be replaced in the proper side pocket.
- Before washing, guide all the straps through the plastic loops and close the Velcro fastenings in order to preserve their adhesive strength over a long period of time.
- After SofTec Genu is washed and air dried, replace the hinged side bars into the proper side pocket. Be sure the colors on the hinged side bars and side pockets match. The upper side bar must be placed in the inner pocket. (Fig. 7).
- Close the side pockets. (Fig. 8).

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

Part of the body this product is used for

In accordance with the indications (knee). See purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Notes on reuse

SofTec Genu is intended to be fitted only once and to one patient.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

SofTec Genu can be disposed of without any restrictions.

Version: 2017-06

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

² A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of supports and orthoses which are relevant to you.

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