

Carbon-prepreg Ankle-Foot Orthosis with enhanced calf stabilization, offering dynamic support and includes two sets of cushioning pads for added comfort.



## SIZE GUIDE

Size	Side	Shoe size (US)	Height	Item-No.
XS	Left	US Women's 3.5 – 5.5	32.5 cm	197R4/L0
S	Left	US Women's 5.5 – 8	33.5 cm	197R4/L1
M	Left	US Women's 8 – 10.5 / US Men's 6.5 – 8.5	34.5 cm	197R4/L2
L	Left	US Men's 8.5 – 11	35.5 cm	197R4/L3
XL	Left	US Men's 11 – 14	37.5 cm	197R4/L4
XS	Right	US Women's 3.5 – 5.5	32.5 cm	197R4/R0
S	Right	US Women's 5.5 – 8	33.5 cm	197R4/R1
M	Right	US Women's 8 – 10.5 / US Men's 6.5 – 8.5	34.5 cm	197R4/R2
L	Right	US Men's 8.5 – 11	35.5 cm	197R4/R3
XL	Right	US Men's 11 – 14	37.5 cm	197R4/R4

The PeroSupport.tec may be customized to foot shape and size by cutting the sole as per requirements. The supplied micro-fleece insole can be cut into shape as well. Carbon-prepreg is not thermo-formable.

The positive effect is only guaranteed if the orthosis is worn together with a shoe.

## INTENDED USE

- ▶ Dynamic readjustment of the foot
- ▶ Restriction of plantar flexion and support of dorsal flexion
- ▶ Prevents lateral ankle twisting (supination position)
- ▶ Prevents side-slips (lateralization)
- ▶ Great energy storage and return by carbon-prepreg material

## INDICATIONS

- ▶ Peroneal paresis
- ▶ Weakness of dorsiflexion of the foot
- ▶ Malalignment of the ankle joint as a result of weakness of dorsiflexion of the foot

## CONTRAINDICATIONS

- ▶ Material interactions/allergies
- ▶ Do not wear directly on injured, irritated, or damaged skin
- ▶ Weight limit: 120 kg
- ▶ Deformation of the foot or ankle
- ▶ Swelling and extension of the calf skin
- ▶ Sensory insensitivity (e.g., diabetic polyneuropathy)
- ▶ Diabetic foot syndrome/ulcer
- ▶ Severe circulatory disorders/congestive edema

