

Topshelf
Manufacturing, LLC
1851 East Paradise Road, Suite A
Tracy, CA 95304
Phone: 866.592.0488 • Fax: 209.834.8832
topshelforthopedics.com

Scan QR Code
for application Video



Tele-Range Elbow Instructions For Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.








Tele-Range Elbow

Instructions for Use

Please read the following instructions completely and carefully before applying this product. Correct application is important to the proper function of the brace.




Indications: Post elbow injury or surgery, range of motion control

Application Instructions:

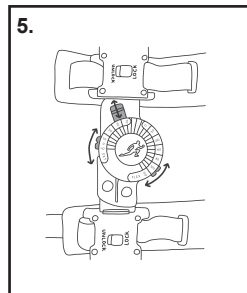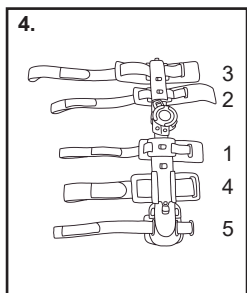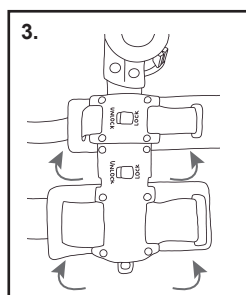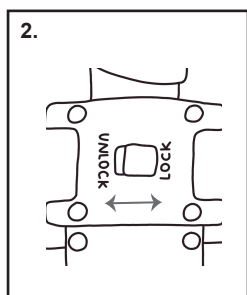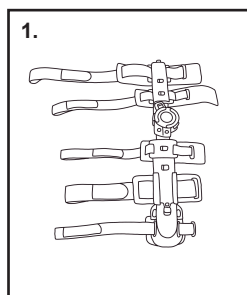

1. Begin by releasing all the straps and positioning the brace on the affected arm with dial hinge over the elbow joint. The brace is right(Rt) or left(Lt) specific.
2. Adjust the length on the upper arm and forearm cuffs by moving the button to the unlock position and sliding to the desired length and then moving the button to lock.
3. Shape the padding around the forearm and upper arm and tighten the length of the straps. If straps are too long they may be shortened by removing the hook and loop end tab and trimming to length then re-affix the hook and loop end tab.
4. Secure the straps closest to the elbow first then the most distal straps by tightening the straps and securing the hook and loop to the strap.
5. Pull the flexion set button away from the center of the circular dial and slide it to the desired angle. The brace can be locked in several positions using the drop lock hinge tab.

To control supination and pronation:

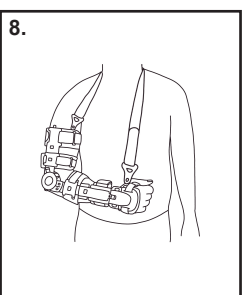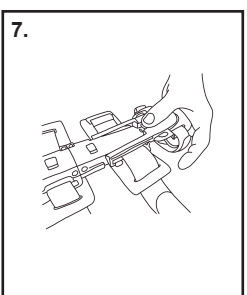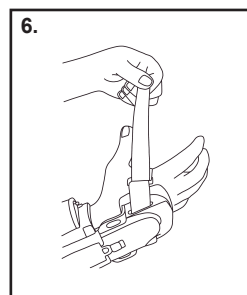

6. The Tele-Range Elbow is packaged with the hand piece attached. When fitting the brace, make sure the hand piece is resting on the posterior side the hand. Wrap the strap around the hand and secure the hook and loop strap. The strap can be shortened by removing the hook and loop end tab and trimming to length then re-affix the hook and loop end tab.
7. To remove the hand piece, push the button and gently pull the hand piece away from the brace.
8. **Optional shoulder strap:** Uncoil the shoulder strap and attach to the upper arm loop, wrap it behind the back and over the shoulder and secure to the forearm loop near the wrist(if no hand piece is present) or to the loop on the hand attachment. The shoulder strap can be shortened by removing the hook and loop end tab trimming to length and the re-affix the hook and loop end tab.

Use and Care: The liner and straps may be hand washed with a mild detergent. Air dry only. Foam and straps may be trimmed and adjusted, but only at the discretion of your healthcare professional

Warning: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product consult your medical professional immediately. All adjustments of the brace should be done under the direction of the healthcare professional.

Warranty: Top Shelf Orthopedics guarantees this product to be free of defects in material or workmanship for a period of six months.

Caution: Federal Law (U.S.A.) restricts the device to sale by or on the order of a licensed health care professional.